DISINFECTANT FOR INSTRUMENTS

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INSTRUCTIONS FOR USE

Product description:

Concentrated detergent, decontaminant and disinfectant solution with a broad spectrum of tuberculous, bactericidal, fungicidal action, and active on viruses with coat, based on quaternary ammonium salts synergised with o-phenylphenol. The phenolic component ensures excellent biocidal activity of the solution in the presence of organic material such as blood, biological fluids, etc.. Specific for hospital surgical instruments. Indications: cleansing, decontamination and disinfection of critical and semi-critical surgical instruments (invasive and non-invasive medical devices) before the sterilization phase.

Methods of Use:

DESINFECTANT FOR INSTRUMENTS VIDU can be used in a free standing tank and ultrasonically by immersing instruments at the established dilution and contact time. Rinse and dry before the next sterilization step.

Dilution and contact time:

DESINFECTANT FOR INSTRUMENTS VIDU is used diluted in water using the dosage chamber integrated in the bottle, for cleaning dilute at 1% (10 ml per liter of water) by immersing the instruments for 10 minutes. For a decontaminating activity dilute at 2% (20 ml per liter of water) by soaking the instruments for 15 minutes.

Chemical composition:

100 g of product contain: benzalkonium chloride 10,0 q; o-phenylphenol 5,0 g; antifoaming agents, corrosion inhibitors, co-formulants, dye and purified water to 100 g. Chemical-Physical properties: appearance: semi-limpid liquid; color: green; odor: slightly alcoholic. THE SOLUTION REMAINS ACTIVE FOR 48 HOURS.

Warnings and precautionary statements:

Product should be handled by trained personnel with appropriate safety precautions. (H)Risk Phrases: H(226): Flammable liquid and vapors; H(314): Causes severe skin burns and eye damage; H(411): Toxic to aquatic life with long lasting effects; H(290): May be corrosive to metals; H(336): May cause drowsiness or dizziness. (P)Cautionary Advice: P(210): keep away from heat, hot surfaces, sparks, open flames or other sources of ignition. Do not smoke; P(234); store only in the original package; P(280): wear protective gloves/protective clothing/eye protection/face protection; P(280): wear protective gloves/protective clothing/eye protection/face protection; P(301/330/331): if swallowed, rinse mouth. Do not induce vomiting; P(304/340): in case of inhalation, remove victim to fresh air and keep at rest in a position comfortable for breathing; P403+P233: Keep container tightly closed and in a well-ventilated place; P(305/351/338): in case of contact with eyes, rinse thoroughly for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing; P(370/378): in case of fire use ABC powder extinguisher. P(403/233): Keep container tightly closed and in a well-ventilated place; P(403+P235): Store in a cool, well-ventilated place; P(501): Dispose of contents/container in accordance with hazardous waste, container or container residue regulations. Keep out of reach of children. Do not use DM mixed with other products. Use DM only in case of intact packaging.

Storage conditions:

Store at room temperature and far from heat sources. Period of validity: 36 months for the intact and correctly stored product. Nature and capacity of containers: 1000 ml closed HDPE bottle with integrated dispenser.) Quality controls: the company operates with a quality system certified UNI EN ISO 9001:2015 - UNI CEI EN 13485:2016. DESINFECTANT FOR INSTRUMENTS VIDU is a class IIb medical device (directive 2007/47/CE, update of directive 93/42/CE, implemented in Italy by D.L.37/2010).

Packaging:

REF: V110120 DISINFECTANT FOR INSTRUMENTS VIDU 1 Lt

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