· Conscious dentistry ·

VIDU

PROPHY PASTE

es · PASTA PROFILÁCTICA
it · PASTA PER PROFILASSI
fr · PÂTE DE PROPHYLAXIE
pt · PASTA PROFILÁCTICA
de · PROPHYLAXE-PASTEN

**"THE FUTURE
IS IN YOUR HANDS"**

VIDU
PROPHY PASTE

en

INSTRUCTIONS FOR USE

PROPHY PASTE VIDU is a universal polishing paste based on Pumice powder. Due to its high RDA value (Radioactive Dentin abrasion) of 237 it has brilliant polishing characteristics. PROPHY PASTE VIDU is applicable inside, as well as outside of the mouth. PROPHY PASTE VIDU does not contain fluoride, because fluoride can damage the bleaching effect.

INDICATIONS

- Cleaning before bleaching
- Professional polishing of teeth
- Polishing of plastics and precious metals
- Polishing of crowns and filling materials

APPLICATION

Open the box and remove the necessary quantity of paste. Close the box immediately. Apply a little quantity of paste on a brush or polishing cup. Do not mix with water. Apply the paste on tooth surfaces or on the materials you want to polish. Work low-speed if it is possible. After the polishing patients have to rinse out thoroughly with water. Clean polished materials with water.

CONTRA-INDICATIONS

Not for use on patients who are known to be allergic.

SIDE EFFECTS

So far no side effects are known. In case of eye contact, immediately rinse with plenty of water. If you experience any undesired effects, even in case of doubt, we ask for notification in every case, describing the concomitants and symptoms as detailed as possible. Processing instructions See under point "application".

WARNINGS/PRECAUTIONS

It is recommended for the patient and the dentist to carry an eye protector. Product use is only indicated for the indication ranges. In case of a preparation for tooth whitening (Bleaching) no paste should remain on the tooth surfaces. This can impair the bleaching effect.

STORAGE

Keep well locked at room temperature (or lower). If sedimentation should happen, or the paste will be drained, then that does not impair the effectiveness of the product.

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Contents: Pumice, Bentonite, Calcium Carbonate and Flavorings.
For professional dental use only. Store in a cool and dry place.

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