PermaFlo® DC Luting and Bonded Post with Primers A&B

Description:

PermaFlo® DC Composite Luting/Bonding Resin in conjunction with PermaFlo® DC Primers A and B, is designed to provide chemical polymerization of the luting resin beneath indirect restorations and endodontic posts which are inaccessible to light. It can be light cured for transparent restorations, i.e. onlays, inlays, and porcelain crowns. It is designed for direct restorations as an initial layer and/or bulk fill for posterior bonded composites and core material prior to crown fabrication.

Indications for Use:

Indirect Crowns, Inlays, Onlays,

- Metal
- Porcelain
- Zirconia

Endodontic Posts

- Fiber
- Ceramic
- Metal

Directions for Use:Indirect Bonding Technique - Indirect Crowns, In-lays, Onlays, Zirconia**1. Prepare**

- Clean preparation, rinse and dry (Consepsis® Scrub with a STARBrush® to clean, disinfect, and remove residual cement).
- Verify prosthetic fit.
- Prepare inside surface of prosthesis.
 - Metal based:
 - Microabrade or sandblast inside surface of prosthesis.
 - Apply phosphoric acid (Ultra-Etch®) to inside surface of prosthesis for 5 seconds to clean, rinse for 5 seconds, dry.
 - Ceramic/porcelain:
 - Apply hydrofluoric acid (Ultradent® Porcelain Etch) to inside surface of prosthesis for 90 seconds, rinse and dry.
 - Apply Ultra-Etch® for 5 seconds to remove porcelain salts and debris formed by HF etching. Rinse and dry.
 - Apply silane (Ultradent® Silane) to inside surface of prosthesis for 1 minute, dry and set prosthesis aside. Do not rinse.
 - Zirconia:
 - Do not etch or use silane, air abrade internal surface of prosthesis.
 - Apply a zirconia primer as per manufacturer instructions.

2. Etch

- Apply Ultra-Etch® with the Blue Micro® Tip to all surfaces of tooth preparation for 20 seconds.
- Rinse thoroughly for 5 seconds and leave damp.

OPTIONAL – For additional antimicrobial effect, apply Consepsis (where available) to preparation with a scrubbing motion and air dry.

3. Bond

- Attached a blue 22ga Deliver-Eze® Brush Tip to the Primer A and Primer B syringes.
- Apply a puddle coat of Primer A and gently agitate for 10 seconds. Do not dry.
- Apply 2-4 drops of Primer B and leave in place for 10 seconds.
- Thoroughly dry 10 seconds using full air pressure. Preparation should appear shiny. If surface does not appear shiny, repeat Step 3a and 3b.

4. Cement

- Remove PermaFlo® DC cap from dual barrel syringe by twisting counter clockwise and discard.
- Prior to attaching tip, verify uniform flow from both syringe orifices by expressing a small amount onto a mixing pad. Use caution not to cross-contaminate.
- Attach mixing tip by lining up internal cap stems with syringe orifices.
Twist clockwise to lock.
- If desired, attach the flexible Intraoral Tip® to the mixing tip for delivery directly into the interior of the prosthesis from the dual barrel syringe.

- e. Apply a thin layer of PermaFlo® DC cement in prosthesis and seat immediately. Working time is approximately 2 minutes 30 seconds.
 - f. Remove excess cement.
 - g. Apply DeOx® (glycerin) to prevent oxygen inhibition formation.
 - h. Light cure margins for 10sec. (20 sec. for lights with output<600mW/cm²).
- NOTE: Set time is 5-8 minutes.

Post and Core

1. **Prepare**
 - a. Drill post hole using a rubber stop and appropriate size UniCore® Drill.
 - b. Thoroughly rinse the post space for 10 seconds using the TriAway™ Adaptor and a 22ga Endo-Eze® Tip. Dry with TriAway Adaptor and 22ga Endo-Eze® Tip from the bottom of the preparation up.
 - c. Verify post (UniCore Post) fit and cut to desired length using a high speed diamond disc. NOTE: Carbide and operative burs may fray the fibers.
2. **Etch-** Always place and remove chemicals from the bottom of the preparation to the top
 - a. Attach a blue 22ga Endo-Eze® Tip to the Ultra-Etch® syringe.
 - b. Start apically and fill post space coronally. Apply to coronal preparation. Etch for 20 seconds.
 - c. Suction off excess etchant with Luer Vacuum Adaptor and 22ga Endo-Eze® Tip.
 - d. Using the TriAway Adaptor and 22ga Endo-Eze Tip, rinse thoroughly and lightly air dry, leaving the post space and coronal preparation slightly damp.
3. **Bond**
 - a. Attach a blue 22ga Deliver-Eze® Brush Tip to the Primer A and Primer B syringes.
 - b. Place 1-2 drops of Primer A into post space and onto preparation, and then agitate full length of the canal including the entire tooth preparation, for 10 seconds. Do not dry.
 - c. Place 1-2 drops of Primer B onto the preparation for 10 seconds.
 - d. Thoroughly remove excess Primer A & B using the TriAway Adaptor and the 22ga Endo-Eze Tip.
 - e. Air dry post space and preparation for 10 seconds using full air pressure to thin/dry with TriAway™ Adaptor and 22ga Endo-Eze® Tip to evaporate any remaining solvent from the bottom up.
4. **Post Cementation and Core Build-up**
 - a. Remove cap from the PermaFlo® DC syringe.
 - b. Place a mixing tip onto the dual barrel syringe lining up the two ports. Twist clockwise until locked in place.
 - c. Attach the flexible Intraoral Tip® to the mixing tip for delivery directly into the canal from the dual barrel syringe.
 - d. Express a small amount before placing in the post space.
 - e. Insert the tip into the post space to full depth.
 - f. Using even pressure deliver PermaFlo® DC starting apically and moving coronally.
 - g. Insert the post (UniCore® Post) slowly, displacing the excess cement. Working time is approximately 2 minutes 30 seconds.
 - h. Place curing light directly above the post and light cure 20 seconds; 40 seconds for lights with output <600mW/cm²)
 - i. Continue building layers of PermaFlo® DC around post as needed. NOTE: Light cure for 5 seconds between layers to gel and form core. Do an final cure on core for 40 seconds. Set time is 5-8 minutes.
5. **Do not remove mixing tip until next use.** Disinfect syringe and mixing tip.

Precautions and warnings:

1. Carefully read and understand all instructions before use.
2. Verify flow prior to applying intraorally. If resistance is met, replace tip and re-check. Use only recommended tips.
3. Dispose of tips and empty syringes properly.
4. Re-cap syringes with Luer Lock cap and disinfect with an intermediate level disinfectant between uses.
5. Resins can be sensitizing. Avoid repeated contact of uncured dental resin with skin. Do not use on patients with a known sensitivity to acrylates or other resins.
If allergic reaction, dermatitis or rash develops, consult a physician.
6. Re-direct operatory light during restorative procedures to avoid premature polymerization.
7. Light activated bonding agents are sensitive to ambient light. Syringe Luer Lock caps should be replaced following use to prevent unwanted polymerization.
Cover syringe tip with gauze if left exposed to ambient light for long periods.
8. Establish adequate isolation.
9. Resins should be refrigerated for long-term storage to maintain shelf life.
10. To avoid cross contamination, do not re-use tips.
11. To optimize bond strength, use oil-free, moisture-free air.

NOTE: For MSDS and additional information about using PermaFlo® DC and PermaFlo® DC Primers A & Band other related products, please go to www.ultradent.com

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
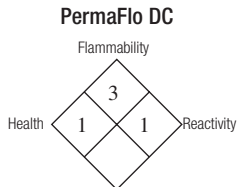
Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

HAZARD RATING

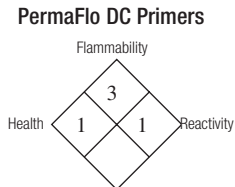4 = Severe
3 = Serious
2 = Moderate
1 = Slight
0 = Minimal

**For product MSDS please visit our
website: www.ultradent.com**

**For product MSDS please visit our
website: www.ultradent.com**

Refrigerate PermaFlo DC if not used on a daily basis.

Refrigerate PermaFlo DC Primers.

For professional use only. Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free: **1-800-552-5512**.

Outside U.S. call (801) 572-4200,
or visit www.ultradent.com.

© Copyright 2011 Ultradent Products, Inc. All Rights Reserved.

 **Manufactured by Ultradent Products, Inc.**
505 West 10200 South, South Jordan, Utah 84095, USA Made in USA

ULTRADENT
PRODUCTS, INC.

#59187.6 102412