· Conscious dentistry ·

VIDU

GUTTAPERCHA POINTS

ES · PUNTAS DE GUTAPERCHA
IT · PUNTE DI GUTTAPERCA
FR · POINTES DE GUTTAPERCHA
PT · PONTAS DE GUTAPERCHA
DE · GUTTAPERCHA STTIFTE

“THE FUTURE
IS IN YOUR HANDS”

VIDU

GUTTA PERCHA POINTS

EN

INSTRUCTIONS FOR USE

1. Brief of the product

The Gutta Percha Points is used for root canal filling and sealing in the treatment of pulpitis. It is sterilized one-time use instruments. It is one-time use instruments and non-sterile instruments.

2. Product Structure and materials

This product is made of Guta gum, zinc oxide, barium sulfate, and a toughening agent (2,6 di-tert-butyl p-cresol).

1) This product has two kinds, one is standard Gutta Percha Points, other one is Great Taper of Gutta Percha Points (04, 06). Standard Gutta Percha Points has the standard size and taper, and it's size is 28±1mm (each mm diameter of length change range within 0.02mm). 04, 06 Great Taper Gutta Percha Points: (each mm diameter of length change range are 0.04mm and 0.06mm) average size is 30±1mm. Gutta Percha Points as uniformity from tip at least 1mm place to 16mm. 2) This product has toughness and hardness, and without excitant root canal organization, meantime with the good resistance function. 3) The Taper of the Gutta Percha Points composition's color should be uniform, smooth, seamless surface, no internal and external foreign body. 4) Gutta Percha Points has good completeness.

3. Scope of application

This product belongs to dental special root canal filling materials, mainly used in the treatment of pulpitis and root canal surrounding tissues in the disease root canal filling.

Warnings

3.1 Only skilled dentists are allowed to use it.
3.2 Be sure to sterilize this product for each use.
3.3 Do not use this product except for the dental service and treatment. Use it in accordance with the intended use.
3.4 Gutta Percha Points cannot be cleaned or sterilized. Cleaning or sterilization may cause the device to become unusable.

Contraindications and prohibitions
None

4. Re-Processing

The instruments are single use and not approved for re-use.

5. Specifications and Method of Use

5.1 The Specification of VIDU Absorbent Gutta Percha Points as the below

(1) Standard taper ISO Types:
15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80, 90, 100, 110, 120, 130, 140

(2) Accessory Type: XXF, FF, MF, F, FM, M, ML, L, XL

(3) Great Taper Type

Great Taper ISO .04: 15, 20, 25, 30, 35, 40, 45, 50, 60, 70, 80

Great Taper ISO .06: 15, 20, 25, 30, 35, 40, 45, 50, 60, 70, 80

5.2 Method of Use:

After uncovering the pulp of the bad tooth and exposing the root canal orifice on the top of the tooth, use tweezers to clamp the stained part of the absorbent paper tip with the same model as the enlarged needle to clean the cavity, and suck out the tissue fluid, pus, blood and other fluids in the root canal.

6. Disinfection and Sterilization before use

This product needs to be disinfected before use or sterilization, recommend the method as following:

1) Soak in MI TA PHENOL DING for 15 minutes, then wash with 75% alcohol and blow dry for standby. 2) Soak in New JIE ER solution for 30 minutes, and then wash with 75% alcohol and blow dry for standby.

This product after physical and chemical properties, biological properties test all reach the standard, so there are no obvious side effects.

7. Storage

This product should be stored in a relative humidity is not more than 80% no corrosive gas, ventilated good indoor

8. Duration of use

In accordance with the storage conditions of the product, since the date of packaging is valid for three years.

9. Disposal

For proper disposal, always observe national laws and recommendations of the authorities.

10. Packaging

Packaging unit: 120pcs/package or 60pcs in a plastic box.

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