## R-080520

E .....	BATIDORA DE VACÍO KOALA .....	4
GB.....	KOALA VACUUM MIXER.....	10
F .....	MALAXEUR SOUS VIDE KOALA .....	14
D .....	KOALA-VAKUUMMISCHGERÄT .....	19
P .....	BATEDORA DE VÁCUO KOALA.....	24
I.....	MESCOLATORE SOTTO VUOTO KOALA .....	29

**MESTRA®**


TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725


mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA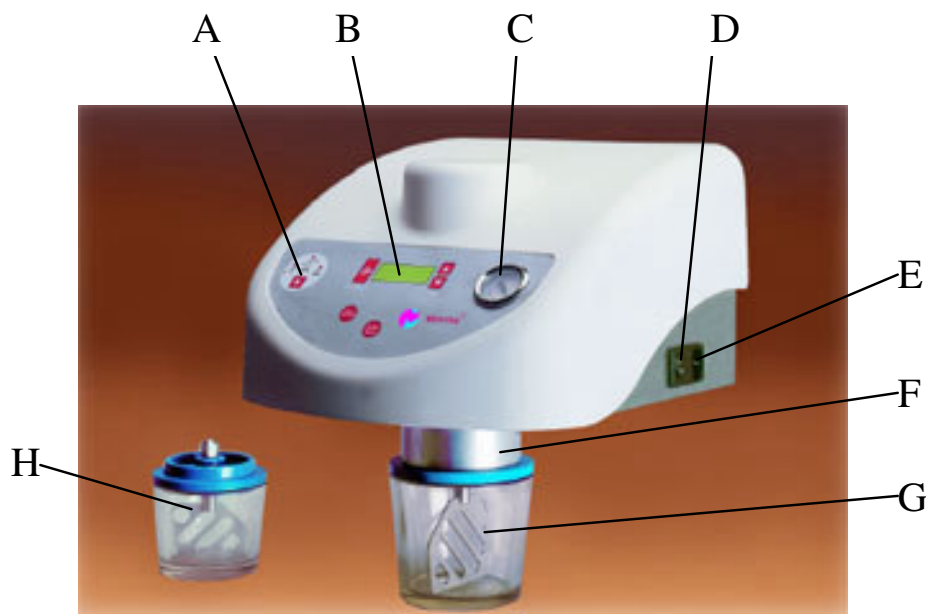

## R-080520

A	Panel de selección modo de funcionamiento / <b>Operation panel board</b> / <i>Panneau de sélection du mode de fonctionnement</i> / Betriebsartwähltafel / <b>Painel de selecção do modo de funcionamento</b> / <i>Pannello di selezione modalità di funzionamento</i>
B	Display electrónico / <b>Electronic display</b> / <i>Affichage digital</i> / Elektronisches display / <b>Display electrónico</b> / <i>Display elettronico</i>
C	Vacuómetro / <b>Vacuum gauge</b> / <i>Indicateur de niveau de vide</i> / Vakuummesser / <b>Vacuómetro</b> / <i>Vacuometro</i>
D	Interruptor general / <b>Master switch</b> / <i>Interrupteur general</i> / Hauptschalter / <b>Interruptor geral</b> / <i>Interruttore generale</i>
E	Toma de corriente con fusible / <b>Electric point with fuse</b> / <i>Prise de courant avec fusible</i> / Stromanschluss mit sicherung / <b>Tomada de corrente com fusível</b> / <i>Presa di corrente con fusibile</i>
F	Boquilla sujeción de taza / <b>Bowl fitting nozzle</b> / <i>Bouche de maintien du bol</i> / Mischbecherhaltekopf / <b>Boquilha de sujeição da tigela</b> / <i>Imboccatura fissaggio tazza</i>
G	Taza grande / <b>Big bowl</b> / <i>Bol grand modèle</i> / Großer mischbecher / <b>Tigela grande</b> / <i>Tazza grande</i>
H	Taza pequeña / <b>Small bowl</b> / <i>Bol petit modèle</i> / Kleiner mischbecher / <b>Tigela pequena</b> / <i>Tazza piccola</i>

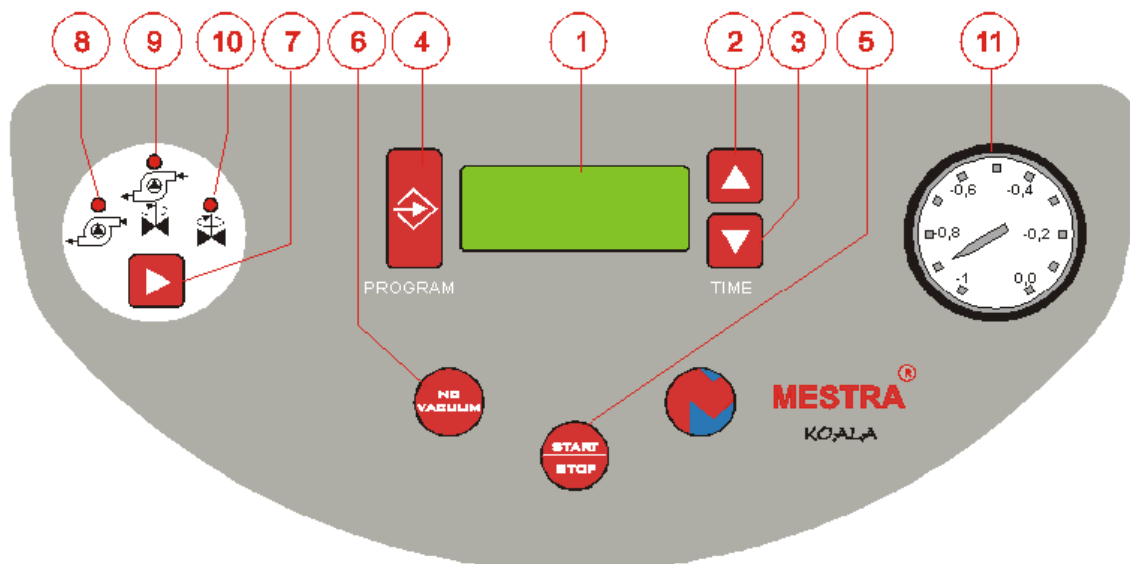1. Display de funcionamiento / **Display** / *Affichage digital* / Display / **Display** / *Display*
2. Tecla “Subir” / **“Up” key** / *Touche + / "Auf"-Taste* / **Tecla “Subir”** / *Tasto “Sollevare”*
3. Tecla “Bajar” / **“Down” key** / *Touche - / "Ab"-Taste* / **Tecla “Descer”** / *Tasto “Abbassare”*
4. Tecla “Programa” / **“Programme” key** / *Touche “Programme” / “Programm”-Taste* / *Tasto “Programma”*
5. Tecla de marcha-paro “Start-stop” / **“Start-stop” key** / *Touche marche-arrêt / "Start-Stop" Taste* / **Tecla marcha-parada** / *Tasto di avvio-arresto “Start-stop”*
6. Tecla de presurización del circuito de vacío (soltar taza) / **Vacuum circuit pressurising key (release bowl)** / *Touche pour pressuriser le circuit de vide (libérer le bol) / Unterdrucksetzungstaste des vakuumkreises (becher lösen) / Tecla de pressurização do circuito de vácuo (soltar a tigela) / Tasto di pressurizzazione del circuito di vuoto (rilasciare tazza)*
7. Tecla de selección del modo / **Operational mode selection key** / *Touche de sélection du mode de fonctionnement / Betriebsartwähltaste / Tecla selecção do modo de funcionamento / Tasto di selezione modalità*
8. Lamparita modo “Sólo vacío” / **“Only Vacuum” mode key** / *Témoin “Vide seulement” / Lämpchen betriebsart "nur Vakuum" / Lâmpada modo “só vácuo” / Lampadina mod. “Solo vuoto”*
9. Lamparita modo “Batido + vacío” / **“Vacuum + Mixing” mode key** / *Témoin “Mélange + vide” / Lämpchen betriebsart "Mischen + Vakuum“ / Lâmpada modo “batido + vácuo” / Lampadina mod. “Mescolatura + vuoto”*
10. Lamparita modo “Sólo batido” / **“Only Mixing” mode key** / *Témoin “Mélange sans vide” / Lämpchen betriebsart "nur Mischen" / Lâmpada modo “só batido” / Lampadina mod. “Solo mescolatura”*
11. Vacuómetro / **Vacuum gauge** / *Indicateur de niveau de vide / Vakuummesser / Vacuómetro / Vacuometro*

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**Koala Vacuum Mixer** (R-080520) makes mixing of investment, plaster and related mixtures easy. This machine makes possible to choose among three different operational modes:

- **“Vacuum” Operational Mode.** With this option, the Mixer will run a vacuum without mixing cycle that will last the time selected on the display. It is a very practical mode when soaking the master models in vacuum atmosphere. Vacuum time can be chosen from 5 seconds up to 99 min. 59 sec. This mode is activated when the “Vacuum” Icon Light is switched on.
- **“Mixing” Operational Mode.** Similar to the previous one, but in this case mixing blades operate while vacuum pump is off. Mixing time can be adjusted from 15 seconds up to 10 min. Every 10 seconds operating in this mode, mixing motor makes a kickback in order to obtain an even mixing. When this mode is option, it is necessary to have a fastening arm (R-080522) that can be purchase as attachment (consult your MESTRA distributor), because as vacuum is not operative, the bowl is not fastened to the mixer nozzle.
- **“Vacuum+Mixing” Operational Mode.** Similar to the previous ones, but in this mode vacuum pump and mixing motor run simultaneously. Time can be chosen from 15 seconds up to 10 min. The machine makes a kickback working this way.

The Mixer also allows programming up to 3 different operational modes for each machine run. For optimum performance and a long lasting life, please read the following instructions.

## INSTALLATION

1. Carefully unpack the KOALA Mixer.
2. Check that in the packaging you find the following accessories:
  - A 250 cc mixing bowl (G).
  - A 550 cc mixing bowl (H).

- A wall anchoring kit.
  - An Allen spanner for fixation flanges.
  - Plastic spatula for plaster-alginate (R-060002).
3. Choose a wall far from heat sources, vibrations or moisture. It is also necessary that this were a firm, rigid and well-levelled surface.
  4. Place the tie down flat bar in the chosen place at a suitable height for working. Make sure this point is horizontal lined perfectly. Then, mark the screw holes on the wall with a marker.
  5. Drill the mentioned marked holes with a hand-held drill and a 10 mm. widia drill.
  6. Fit the nylon rawlplug plug included in the mixer anchoring kit. Place the tie down flat bar on the wall and fix it firmly using the screws from the kit.
  7. Mark on the wall 2 vertical lines at 75 mm from each end of the tie down flat bar. These marks will help you to centre the Mixer.
  8. Remove the 2 fixation flanges from the back lower side of the machine. Place the machine on the tie down flat bar. Make sure it is well centred taking into account the marks done. Put the fixation flanges again. Make sure flanges are correctly placed on the tie down flat bar.
  9. Thread the flanges screws till the machine is securely fastened.
  10. Connect the Mixer mains cable to 230 V, 50/60 Hz equipped with earth return.

## HOW TO USE THE MIXER

Once the installation is finished, the Mixer is ready to be used:

1. Activate the Mixer master switch (D). After some seconds the indicator light of the last selected mode will light and the display (1) will show the last programmed time.
2. Press the selection key (7) to choose the machine operation mode. With each pulsation, you will find that the light corresponding to each of the modes will light (8), (9) and (10) (Mixing, Vacuum and Vacuum+Mixing).

3. Once the mode wanted is selected (the corresponding icon light lights) introduce the cycle time pressing Up/Down keys (2) and (3) till the minutes (left two digits) and seconds (right two digits) required appear on the display. Control internal memory storages the last programmed cycle time and mode until they are changed.
4. Select between the two available bowl capacities the one that best suits to the mixing quantity that is to be made. Pour in the bowl the products to be mixed, following the proportion recommended by manufacturer.
5. Make a prebeating of the products in the same bowl using a hand spatula.
6. Place the bowl on the Mixer Bowl Fitting Nozzle (F) pressing a bit to correctly place the joint and press the Start/Stop key (5). The Mixer will start operating.
7. Check by the Vacuum Gauge (11) that vacuum is made in the bowl. Otherwise, slightly move it to place it correctly. Hold the bowl till vacuum rises above 50%. Then let go of the bowl, the mixer will run the cycle showing at each step in the display how long it will take to the end.
8. In case the machine is only in "Mixing" mode, you will have to hold the bowl to the nozzle by a fastening arm (R-080522).
9. Once the cycle finishes, the machine emits an acoustic signal, "End" appears written on the display and it stops. Then press "No Vacuum" Key (6) in order to pressurize the vacuum circuit and be able to take down the bowl. The mixture is ready to be used.

While the cycle is running, it can be cancelled pressing Start/Stop key (5) for three seconds. After this, the Mixer stops, makes an acoustic signal and goes back to stand-by mode, cancelling this way the cycle.

In order to avoid accidents and fall of the bowl, vacuum circuit pressurizing key "No Vacuum" (6) is blocked when the Mixer is running. This key can only be pressed once the machine is in Stand-by mode (cycle end or cancelled cycle).

If there is a current breaking longer than 5-10 seconds when the Mixer is running a normal cycle, once the supply returns, the

display will show "Err" and emit an acoustic signal till any key is pressed. This is meant not to damage the motor in a later starting up when the breaking has lasted the time needed for the mixture to set. When this happens, we recommend throwing away the mixture and preparing a new one.

### **MIXER PROGRAMMING**

KOALA Vacuum Mixer (R-080520) allows up to 10 programmes memory storage. Each of these programmes can have 3 steps maximum. This way, we can programme for instance a cycle beginning with a "Vacuum" cycle for a given time, then the machine can continue with a "Vacuum+Mixing" for the required time and can end up with another "Vacuum" step. Furthermore, it is possible to launch two programme types:

- Direct Programmes: Cycles are run one after another without stopping after each step.
- Programmes with stop: The Mixer stops after each cycle and waits till the "Start/Stop" key is pressed to continue with the next cycle. When a programme with stop is set, the stop can last 5 min. maximum, time when the user can remove the bowl after pressing "No Vacuum" key and handle it. After this time, the user should place the bowl again and press "Start/Stop" key, and the machine will continue the cycle. If the stop is longer than 5 minutes, the control cancels the programme, making an acoustic signal and the display will show "Err" for 15 seconds. After that, it goes back to stand-by.

### **EDITING A PROGRAMME**

In order to edit a programme the machine has to be in stand by mode and then press the Programme Key (4). The display will show the last programme run. Choose by Up/Down Keys (2) and (3) till required programme number appears on the display. Once it is chosen, follow the instructions:

1. Press the Programme Key (4) again for approx 3 seconds. You may see on the display "PX-1", this mean we are programming X programme in step 1. If "End" is showed alternatively means that

the other steps are empty and they will not run.

2. Select Operating Mode (Mixing, Vacuum and Vacuum+Mixing) pressing Selection Key (7).
3. Select wanted time pressing Up/Down Keys (2) and (3). If there is no Operating Mode selected previously, Up/Down Keys will block.
4. If you want that after running this step the machine stops (Programme with stop), press Start/Stop Key. The light corresponding to Operating Mode (8), (9) or (10) will light off and on quickly, indicating that after this cycle the programme will pause. To erase the pause, press "Start/Stop" Key and the corresponding light will stop flickering.
5. Press Programme Key to memorize the cycle and continue with the next one. The display will show "PX-2". Repeat previous steps to finish cycles "PX-2" and "PX-3".
6. When we programme a cycle, for instance No. 1, and we want to finish and conclude the programme, we just have to press Programme Key for 3 seconds. The rest of cycles will erase finishing this way edition, showing the display "End".

It is possible to run at any moment programmes stored in memory. For this, when the machine is in stand-by:

1. Press "Programme" Key (4). The display will show the last run programme.
2. Select with "Up/Down" Keys (2) and (3) the programme number that you want to be run.
3. Press "Start/Stop" Key (5). The machine will run the selected programme.

## PROGRAMMING EXAMPLES

### Example 1

Suppose you want to edit Programme No. 3 so that the following cycle runs:

- An initial step of only "Vacuum" for 10 seconds.
- A second step of "Vacuum + Mixing" for 1 minute.

This programme example has an interesting practical use that allows fixing the bowl to the mixer some seconds before the mixing, so operation is more convenient.

1. Press "Programme" Key. The display will show the number of the last run programme.
2. Press "Up/Down" Keys (2) and (3) until "Pr-3" is showed.
3. Press "Programme" Key again for 3 seconds. You will know you are in programming mode because the display shows "P3-1" flickering.
4. Press "Selection" Key till the "Vacuum" Mode light lights.
5. Select 10 seconds with "Up/Down" Keys.
6. Press "Programme" Key. "P3-2" will appear on the display indicating we are in Programme 3 step 2.
7. Press "Selection" Key till "Vacuum + Mixing" Mode light lights.
8. Select 1 minute with "Up/Down" Keys.
9. Press "Programme" Key for approx. 3 seconds." End" will appear on the display and the machine would go back to stand-by mode. The programme is now stored in memory.

### Example 2

Suppose you want to edit Programme No. 5 so that the following cycle runs:

- An initial step of only "Mixing" for 45 seconds. After this step, the Mixer stops so that the user can handle the mixture. Then, the user should place the bowl again and press "Start/Stop" key, and the machine will continue the cycle.
- A second step of "Vacuum" for 10 seconds. This makes easier placing the bowl on the Mixer.
- A third step of "Mixing + Vacuum" for 1 minute. Once it finishes the cycle is ends.

Programming sequence should be as follows:

1. Press "Programme" Key. The display will show the number of the last run programme.
2. Press "Up/Down" Keys (2) and (3) until "Pr-5" is showed.
3. Press "Programme" Key again for 3 seconds. You will know you are in programming mode because the display shows "P5-1" flickering.
4. Press "Selection" Key till the "Mixing" Mode light lights.
5. Select 45 seconds with "Up/Down" Keys.

6. Press "Start/Stop" Key to indicate we want a pause after this step. You will see "Mixing" light mode flickering quickly.
7. Press "Programme" Key. "P5-2" will appear on the display indicating we are in Programme 5 step 2.
8. Press "Selection" Key till "Vacuum" Mode light lights.
9. Select 10 seconds with "Up/Down" Keys.
10. Press "Programme" Key for approx. 3 seconds."P5-3" will appear on the display indicating you are in Programme 5 step
11. Press selection key until "Vacuum + Mixing" light lights.
12. Select 1 minute with "Up/Down" Keys.
13. Press "Programme" Key for approx. 3 seconds."End" will appear on the display and the machine would go back to stand-by mode. The programme is now stored in memory.

### MAINTENANCE AND CLEANING

To ensure an optimum reliability and long life of the machine, we recommend:

- Carefully clean the mixing bowls after using them. Avoid dirt accumulation and possible blade shaft seizing. Make sure to clean the joint and fitting of the bowl to get optimum sealing.
- Cleaning of external body of the machine should be made with a soapy water dampened cloth. Do not use solvents or any other flammable product.
- Change the filtering element periodically by turning anticlockwise a quarter turn the filter glass on the backside of the machine. Change the filtering element and place the glass on its place again.
- In case there were a current breaking while the cycle is running, the bowl will remain at the nozzle because of vacuum. Under these conditions it may happen that the mixture sets. To avoid this inconvenience, you may see that there is a drain-valve at the bottom of the glass. Pressing the drain-valve shaft the vacuum in the vacuum circuit pressurizes and the bowls gets free. **Carry out this operation only in case of emergency.**
- It is recommended to change mixer and bowl rubber joints periodically. Vacuum in vacuum circuit will be optimised.

- We recommend you to use always Mestra original spare parts in Mixer servicing.

### PRECAUTIONS

- ◆ Before connecting the machine, make sure it is a 230 V, 50/60 Hz electric point with earth.
- ◆ Never place the bowl when the Mixer is running.
- ◆ Do not let children or not qualified people handle the machine.
- ◆ Choose a firm, rigid and well-levelled wall to place the Mixer. Make sure rawlplugs are well fixed in the wall.
- ◆ After a cycle has finished wait till the display shows "End" to remove the bowl.
- ◆ While cleaning operations disconnect the machine from electricity supply.
- ◆ Use the drain-valve to pressurize the vacuum circuit just in case of emergency, (current breaking while the cycle is running).

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