BASEPLATE

en - Light-curing composite material for the fabrication of bite templates, ready-to-use
es - Material compuesto fotopolimerizable para la fabricación de plantillas de mordida, listo para su uso
it - Materiale composto fotopolimerizzabile per la realizzazione di mascherine da morso, pronto per l'uso
fr - Matériau composite photopolymérisable pour la fabrication de gabarits d'occlusion, prête à l'emploi
pt - Material composto fotopolimerizável para a fabricação de moldes de mordida, pronto para uso
de - Lichthärtendes Kompositmaterial zur Herstellung von Bisschablonen, gebrauchsfertig

"THE FUTURE IS IN YOUR HANDS"

Patient target group
 All persons for whom individual trays are required for treatment.
Intended user
 Dentist
Composition Qualitative data
 Oligomeric urethane acrylate, vinyl ester resin, 2,2-dimethoxy-1,2-diphenylethane-1-one or by 2,2-dimethoxy-2-phenylacetophenone, (+)-2,3-dioxo-1,7,7-trimethyl-bicyclo-(2,2,1)-heptane di-bornane-2,3 dione, diphenyl(2,4,6-tri-methylbenzoyl)phosphinoxide, 2-hydroxy-2-methyl-1-phenyl-propane-1-one, 2-ethylhexyl-4-(dimethylamino)benzoate, 2,2-diethoxyacetophenone, soda-lime glass, amorphous silica, dyes (Pigment Red 146 = CI 12485 / Pigment Blue 29 = CI 77007 / D&C Red 30 = CI 73360 / Pigment Blue CI 74160 / Pigment Violet 19 = C.I. 73900), flavourings (Optamint).
Quantitative data
 In 100 g there are 19 g acrylates/methacrylates and 80 g glass. Intended purpose BASEPLATE VIDU is a light-curing tray material in the form of prefabricated templates for the fabrication of torsion-resistant customized impression trays. Alternatively available thinner material (base plates) can be used for the fabrication of bite templates, bite registrations and try-ins.
Indication
 Tray material:
 - Customised trays
 Base plates:
 - Bite registration
 - Try-in
Contraindications
 The material contains methacrylates.
 Do not use in patients with an allergy to methacrylates or other substances contained in the product.
Performance characteristics of the product
 BASEPLATE VIDU polymerises with light to form a solid and compact polymer.
Specific precautions
 Avoid any contact between the eyes and skin and the uncured material, i.e. also with the inhibited layer.
 Always use a dust extractor when trimming the tray!
Preparation/Manufacture
 - Mark the outline of the future tray on the model.
 - Block out the undercuts on the model.
 - To create a spacer for the impression material, block out the entire model (e.g. with a sheet of wax) and cut out stops.
 - If the plaster is very dry, soak the model briefly in water or apply dye isolation or an alginate isolation.
 - Remove the baseplate from the box and close it.
 - Once the model has been blocked out, adapt the baseplate to it and cut to shape using, e.g. a scalpel.
 - Use the remaining material to form a handle and attach it to the tray.
 - Place the model and tray in a light-curing unit and cure without vacuum.
 - Remove the tray from the model and polymerize once again from the underside.
 - If the tray has been cured without a vacuum, use alcohol to remove the smear layer.
Times
 Working at daylight 20 minutes
 Polymerisation in an ultraviolet or LED light-curing units, depending on the unit in use:
 1. The top of the tray, on the model 3-5 minutes
 2. The underside of the tray, off the model 3-5 minutes
Trimming
 - Trim the custom tray with tungsten carbide burs and smooth it with sandpaper.
Hygiene
 Immerse impressions taken with BASEPLATE VIDU customs trays in a disinfectant solution based on glutar aldehyde. The time depends on the disinfectant manufacturer's directions. The finished medical product is used by the dentist, e.g. for individual trays with impression material.
Caution
 The product has been developed for use in the dental sector and must be used according to the directions. The manufacturer accepts no liability for damage that results from use other than the above. Furthermore the user is obliged under his own responsibility to examine the material for its suitability and its usage possibilities for the intended applications, especially if the latter are not detailed in the directions for use.
Side effects
 Adverse side effects are not known.
Interactions
 Interaction of the product with other medical devices, medicinal products or other substances is not known.
Storage
 The Product should be stored between 0° C - 22°C
Technical data
 Flexural strength min. 130 N/mm2
Minimum shelf life
 Do not use BASEPLATE VIDU after the expiry date.
Variants
[\[REF\]](#) V179010 pink, 50 pcs
Disposal
Uncured material can be polymerised with light (sunlight or artificial light) and then disposed of as plastic waste. All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

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