· Conscious dentistry ·

VIDU

DESINFECTANT DETERGENT

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en

INSTRUCTIONS FOR USE

SURFACE DISINFECTANT FOR NON-INVASIVE MEDICAL DEVICES

Tuberculocidal, bactericidal, mycobactericidal, fungicidal and active against coated viruses

Product description:

Disinfectant based on isopropyl alcohol and o-phenylphenol. This combination provides excellent broad spectrum disinfectant activity: fungicidal, bactericidal, mycobactericidal and active on coated viruses. It also has excellent detergent and solvent properties on organic residues, such as blood and biological fluids.

Indications:

Product indicated for disinfection and decontamination of non-invasive medical devices present in the operating room.

Directions for use:

Spray directly on the surfaces or devices to be treated.

Contact time:

In clean conditions it is bactericidal in 1 minute and active on coated viruses in 5 minutes. In extremely dirty conditions active on fungi and bacteria in 5 minutes, active on coated viruses (HIV, HBV, HCV) in 10 minutes, active on mycobacteria (TBC) in 15 minutes. Then dry with a clean cloth or paper.

Chemical composition:

100 g of product contains: alcohol mixture 30 g, 0-phenylphenol 0,05 g, DDQ 0,1 g, nonionic surfactants, sequestering agents and purified water up to 100 g. Chemical and physical characteristics: Appearance: clear liquid; color: colorless; odor: characteristic.

Warnings and cautionary statements:

(H)Risk phrases: H(226): flammable liquid and vapors. H(319): irritant to eyes. H(336): may cause drowsiness or dizziness. (P)Precautionary statements: P(210): keep away from heat, hot surfaces, sparks, open flames or other ignition sources. Do not smoke. P(280): Wear gloves. P(304/430): In case of inhalation, remove victim to fresh air and keep at rest in a position to facilitate breathing. P(305/351/338): In case of contact with eyes, rinse thoroughly for several minutes. Remove any contact lenses, if comfortable. Continue rinsing. P(370/378): in case of fire, use an ABC powder extinguisher to extinguish. P(403/233): Keep container tightly closed and in a well-ventilated area. P(403/235): Store in a cool, well-ventilated place. P(501): dispose of product/container in accordance with regulations on hazardous waste, packaging or packaging waste. Keep out of the reach of children. Do not use DM if the container is not intact. Do not use DM mixed with other products.

Storage Modality:

Store at room temperature and away from heat sources. Shelf life: 36 months for intact and properly stored product. Nature and capacity of containers: Closed 750 or 5000 ml HDPE can with sealed lid. Quality controls: the company operates under a UNI EN ISO 9001:2015 - UNI CEI EN 13485:2016. DISINFECTANT DETERGENT VIDU is a medical device of IIa class (Directive 2007/47/EC, update of Directive 93/42/EC, implemented in Italy by Legislative Decree 37/2010).

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