

CLEANMED INSTRUMENTS



 - 1L Bottle



 - 5L Container

PRODUCT DESCRIPTION

Disinfecting concentrate for instruments, burs and endodontic instruments and items.

Instruments disinfection is one of the main daily activities and one of the most vital for the patient protection. The basic requirements for a strong instrument disinfectant are material compatibility, a broad effective spectrum and anti corrosion inhibitors. Our products have been tested on a wide variety of materials for product compatibility.

Main characteristics:

- Aldehyde and Phenol free;
- 99% biodegradable;
- Effective against a wide spectrum of pathogens;
- Easy and accurate dosage system for different % solution;
- Geranium Aroma;

COMPOSITION

Aqua, Benzalkonium Chloride, Didecylidimethylammonium Chloride, Propan-2-ol, Benzotriazole, Polyhexamethylene Biguanide Hydrochloride, Hydroxyethylcellulose, Cocamidopropyl betaine, Parfum, C.I. 44045.

Product name	Code
Cleanmed Instruments 1L	7.005-1L
Cleanmed Instruments 5L	7.005-5L
Dispenser Tap for 5L Container	7.00D



STANDARDS APPLIED:

- ✓ EN 14885: 2015
- ✓ EN 13727: 2012 + A1:2013
- ✓ EN 14476: 2013
- ✓ EN 13624: 2013
- ✓ EN 14561: 2006
- ✓ EN 14562: 2006



EFFECTIVE SPECTRUM

BACTERICIDAL	Contact Time
Pseudomonas aeruginosa	15 min
Enterococcus Hirae	15 min
Staphylococcus Aureus	15 min
Salmonella Typhimurium	5 min
Legionella Pneumophila	5 min
YEASTICIDAL	
Candida Albicans	15 min
VIRUCIDAL	
HIV1	5 min
Hepatitis C Virus	5 min
Humana Influenza B Virus	2,5 min



SOLUTION

Concentrated to be diluted at 2%



WARNINGS

- Do not exceed the **soaking time of 15 minutes.**
- Do not use on alloyed materials, malleable iron or aluminium casting parts, e.g. citoject syringe.