

## CLEANMED SUCTION



- Bottiglia dosatrice 1L

- Container 5L

### PRODUCT DESCRIPTION

Concentrated disinfectant for cleaning and daily maintenance of suction systems.

*The suction systems are continuously exposed to pathogens and represent a danger to both the dental staff and patients, as well as to the functionality of the suction system. Cleaning and disinfection should occur frequently, at least once a day or at the end of each surgery. Cleanmed Suction is specially formulated to be compatible with suction systems and to have a broad effective spectrum, in order to preserve the functionality of the system and ensure its hygiene.*

### Main features:

- Chlorine, Aldehyde and Phenol free;
- Alkaline-based solution;
- Non-foaming and non-abrasive;
- Easy and accurate dosage system for different % solution;
- Effective also for the spittoon bowl;

### COMPOSITION

Dimethyldiocyammonium chloride, dipropylene glycol, 2-aminoethanol, ethylenediamine tetrasodium tetraacetate, ethanol, citric acid, non-ionic surfactant, defoamer, dye.

Product Name	Code
Cleanmed Suction 1L	7.006-1L
Cleanmed Suction5L	7.006-5L
Dispenser Tap for 5L container	7.00D



### STANDARDS APPLIED:

- ✓ EN 14885: 2015
- ✓ EN 13727: 2012 + A1:2013
- ✓ EN 14476: 2013
- ✓ EN 13624: 2013
- ✓ EN 14561: 2006
- ✓ EN 14562: 2006



### EFFECTIVE SPECTRUM

BACTERICIDAL	Contact Time
E-Coli	15 min
Pseudomonas aeruginosa	15 min
Enterococcus Hirae	15 min
Staphylococcus Aureus	15 min
Salmonella Typhimurium	15 min
Legionella Pneumophila	15 min
TUBERCULICIDAL	
Mycobacterium Terrae	60 min
YEASTICIDAL	
Candida Albicans	15 min
VIRUCIDAL	
HIV1	15 min
Hepatitis C Virus	15 min
Humana Influenza B Virus	15 min
FUNGICIDAL	
Aspergillus Brasiliensis	30 min



### SOLUTION

Concentrated to be diluted at 2%



### WARNINGS

- Do not use on aluminum or cooper parts.