· Conscious dentistry ·

FLOWABLE COMPOSITE

- en** · Light curing small hybride-composite (all purpose)
es · Composite fluido micro-hibrido fotopolimerizable
it · Fotopolimerizzabile per piccole superfici di composito ibrido
fr · Composite hybride photopolymérisable, fluide (por tous usage)
pt · Composito micro-hibrido flúvel fotopolimerizável
de · Lichthärtendes Feinglas-Hybridcomposit (universell einsetzbar)

“THE FUTURE
IS IN YOUR HANDS”

H317 May cause allergic skin reactions
H318 Causes serious eye damage

Danger
Danger

VIDU

FLOWABLE COMPOSITE VIDU

INSTRUCTIONS FOR USE

Patient target group: Children, adolescents, adults and older patients

Intended user: Dentist

Composition: Qualitative composition:

1,4-butanediol dimethacrylate, bis-GMA, aliphatic urethane dimethacrylate, butylhydroxytoluene, camphorquinone, silicon dioxide, glass ceramic, iron oxide pigments, 2-(2H-benzotriazol-2-yl)-p-cresol, 2-ethylhexyl 4-(dimethylamino)benzoate, diphenyl(2,4,6-trimethylbenzoyl)phosphine oxide

Quantitative composition:

1,4-butanediol dimethacrylate 5%, aliphatic urethane dimethacrylate 22%, bis-GMA 18%, silicon dioxide 6%, activators and UV absorbers <1%, glass ceramic 48%, pigments <1%

Intended purpose of the product: FLOWABLE COMPOSITE VIDU is a light-curing, radiopaque, flowable composite filling material for the dental restoration of Class III, IV and V cavities in children, adolescents, adults and elderly patients by a dentist.

Indications: For fillings of cavity classes III, IV and V.

Contraindications: Do not use on patients with known allergies to the ingredients. Do not use directly on the opened pulp.

Use is contraindicated if draining or the prescribed application technique is not possible.

Performance characteristics of the product: FLOWABLE COMPOSITE VIDU is a flowable composite that cures with light.

Specific precautions: None

Application:

- Clean and dry the preparation. Determine the colour.
- Prepare the cavity according to general adhesive technique rules. Bevel the enamel edges.
- Etch the enamel margins with etching gel for 20 to 35 seconds. 4. Rinse off etching gel completely.
- Dry with oil-free air. Avoid overdrying the dentine.
- Apply adhesive according to the instructions for use.
- Select the syringe in the desired colour. Remove syringe cap. Screw on the Luer-Lock needle.
- FLOWABLE COMPOSITE VIDU can be applied directly by pressing the syringe. To ensure complete hardening, apply FLOWABLE COMPOSITE VIDU in layers no thicker than 2 mm.
- Light-cure each layer for 30 to 40 seconds. The halogen or LED light curing device should emit at 450 nm with an illumination intensity $\geq 600 \text{ mW/cm}^2$ and be checked regularly. Position the light as close as possible to the filling material. Do not remove the oxygen-inhibited layer, as it is required for bonding with the next layer. The use of a mould is recommended.
- Repeat steps 8 and 9 until the cavity is filled.
- Excess removal, contouring and finishing are carried out using diamond finishers and flexible discs with decreasing grain size.

Precautions

The user and patient should wear protective goggles before application, as well as when applying etching gel. Avoid eye contact.

In case of eye contact, rinse immediately with plenty of water and seek medical advice.

Avoid contact with the skin.

In the event of skin contact, remove with a cloth and alcohol and then wash thoroughly with soap and water. In case of hypersensitivity of the skin or skin rash, discontinue use and consult a doctor.

The material contains titanium dioxide and glass ceramic, and it should only be ground outside the mouth under an intact fume hood.

Details: Delivery only to dentists and dental technicians or on their behalf. Keep out of the reach of children! Only dispose of completely emptied packaging for recycling.

Side effects: No undesirable side effects are known.

Interactions: Avoid materials containing eugenol, moisture and oily air, as these can prevent polymerisation at the point of contact.

Discolouration may occur if cationic mouthwashes, plaque disclosing agents or chlorhexidine are used.

Storage: Do not expose to direct sunlight or other heat sources. Store in a dry place.

The product should be stored between 0°C to 22°C.

Minimum shelf life: 3 years. Do not use after the expiry date.

Variants:

- REF VI92101 1 x syringe A1 with 1,0 ml + cannulas
REF VI92102 1 x syringe A2 with 1,0 ml + cannulas
REF VI92103 1 x syringe A3 with 1,0 ml + cannulas
REF VI92104 1 x syringe A3,5 with 1,0 ml + cannulas
REF VI921051 x syringe B1 with 1,0 ml + cannulas
REF VI92106 1 x syringe B2 with 1,0 ml + cannulas
REF VI92108 1 x syringe C2 with 1,0 ml + cannulas
REF VI92120 3 x syringe (A2, A3, A3,5), 1,0 ml

Disposal: Any unused material can be disposed of in household waste after curing. All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

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