Instructions for use
Patient target group Children, adolescents, adults and older patients Intended users Dentist Composition Qualitative data: Oligomeric methacrylates, HEMA, camphorquinone Quantitative data in % G/G: Methacrylates 99.5 Additives 0.5 Intended purpose of the product DENTAL BONDING Self EtcH VIDU is a self-etching, light-curing one-component Adhesion promoter for the adhesive restoration technique. Indications Adhesion promoter for the adhesive restoration technique. - Attachment of composite materials and compomers to enamel and dentin - Adhesive fixation of composite restorations on enamel and dentin - Fixing of composite material, metal or amalgam - Also suitable as a dentin sealant Contraindications Phenolic or other substances (e.g. zinc oxide eugenol), which could influence the polymerisation, must not be mixed with the DENTAL BONDING Self EtcH VIDU. Performance characteristics of the product Curing by irradiation with light. Specific precautions None Application 1. Conventional filling technique a) Cavity Preparation According to the principle of adhesive restoration technique. If necessary, a punctiform overlapping of the pulp with a hard-setting calcium hydroxide cement should be laid. Processing of enamel with a diamond grinder. Etch unpolished enamel with phosphoric acid or clean with pumice stone. After that, rinse the enamel well and dry it. The use of a rubber dam is recommended for hygienic reasons. b) Application Shake the bottle well before use and add a drop to the spot shape. Massage well with a disposable brush DENTAL BONDING Self EtcH VIDU 20 s on dentin and enamel. Gently blow the DENTAL BONDING Self EtcH VIDU 5 s with oil-free compressed air. Then 10 s light curing (halogen or LED polymerization devices with a light output >800 mW/cm3). With lower light output, light curing is prolonged accordingly. c) Composite and composite fillings Apply restoration material according to the respective manufacturer's instructions. 2. Fixing of inlays/onlays, veneers and crowns When using DENTAL BONDING Self EtcH VIDU, make sure that sufficient light (according to the manufacturer's instructions) is supplied to the complete Restoration can take place. a) Preparation of the restoration Pre-treatment of the Restoration is carried out according to the material and according to the respective manufacturer's instructions. b) Preparation of the cavity Proceed as described in point 1.1. c) Application Proceed as described in point 1.2. d) Cement According to the respective manufacturer's specifications. 3. Bonding of other surfaces a) Preparation b) Metal and amalgam Roughen or sandblast surfaces with a diamond grinder. Then clean with steam jet (extraoral) and acetone and dry with oil-free air. c) Composite Roughen composite surfaces with a diamond grinder or sandblast, clean with phosphoric acid, rinse and dry with oil-free compressed air. d) Application Shake the bottle well before use and dispense a drop into the spot mould. Close the bottle again after use. Massage DENTAL BONDING Self EtcH VIDU well on the surface with a disposable brush for 20 s. Using oil-free compressed air, gently blow off the bonding during 5 s. Then 10 s lightcuring (halogen or LED polymerisation devices with a light output >800 mW/cm3). With low light output, light curing is prolonged accordingly. e) Restoration Apply restoration material according to the respective manufacturer's instructions. 4. Sealing dentin Proceed as described in points 1.1 and 1.2. Working instructions The working instructions and precautionary measures described in the package leaflet accompanying the product must be observed in any case. Should an undesirable reaction occur in the patient despite compliance or other undesirable events occur, we ask in any case for notification with as precise a description of the accompanying circumstances as possible. Hygiene tips Compliance with the following instructions ensures optimal hygienic conditions when using DENTAL BONDING Self EtcH VIDU: - Use disposable brushes on one patient at a time - After use, close the bottles with the cap and disinfect with common means. Warnings Riskal risks Emergency measures, symptoms, counter-measure An acute risk to the patient due to accidental ingestion should be excluded in view of the minimum quantities to be applied (application with a disposable brush). In case of direct contact with the oral mucosa, rinsing is sufficient. In case of eye contact, rinse thoroughly with water. An ophthalmologist should be consulted for evaluation and further treatment. Toxicological effect The toxicity of the DENTAL BONDING Self EtcH VIDU directly touching the vital dentin and penetrating into the dentin tubules as intended is comparable to glass ionomer cement or polycarboxylate. Details Delivery only to dentists and dental technicians or on their behalf. Keep out of the reach of children! Only completely emptied Packaging and recycling. Side effects Undesirable side effects of this medical device are to be expected with proper use extre- mely rarely. Should you become aware of any undesirable side effects – even in doubtful cases – please inform us in any case, describing the accompanying circumstances and symptoms as precisely as possible. Components of DENTAL BONDING Self EtcH VIDU may cause sensitisation in predisposed individuals. Risks in case of overdose are not known. Interactions Substances containing phenol or other substances (e.g. zinc oxide eugenol) that could influence the polymerisation must not be brought into contact with the DENTAL BONDING Self EtcH VIDU Storage Do not expose DENTAL BONDING Self EtcH VIDU to direct sunlight or other heat sources. The product should be stored between 0°C -22°C. Minimum shelf life 3 years Do not use after the expiry date Variants REF V91003 DENTAL BONDING Self EtcH VIDU – 5ml Disposal Unused material can be cured and sent to household waste. Uncured material should be disposed of like methacrylate waste. All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

“THE FUTURE IS IN YOUR HANDS”

	H317 may cause allergic skin reactions

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