· Conscious dentistry ·

INDICATOR TAPE

en · Chemical indicator tape for steam sterilization
es · Cinta indicadora química para la esterilización por vapor
it · Striscia di segnalazione chimica per la sterilizzazione a vapore
fr · Bande indicatrice chimique pour la stérilisation à la vapeur
pt · Banda indicadora química para esterilização a vapor

"THE FUTURE
IS IN YOUR HANDS"

VIDU

en

INSTRUCTIONS FOR USE

INDICATOR TAPE VIDU is a chemical indicator tape for steam sterilization consisting of a non-cellulosic plastic backing with a pressure-sensitive adhesive on one side and chemical indicator stripes on the other side.
The chemical indicator stripes change from yellow to black after exposure to steam.

UNPROCESSED

FAILURES

PROCESSED AND SATISFACTORY

INSTRUCTIONS FOR USE

- Use the indicator tape on the outside of each package to show that packages have been closed and exposed to a sterilization process, and to differentiate between processed and unprocessed packages.
- Place one end of the indicator tape on the package and press with your finger along the tape to ensure closure and adhesion. Do not stretch the tape as it may tear.
- The tape is suitable for use on non-woven disposable wraps and pouches.
- After sterilizing the load the tape will change color from beige to black.
- When you are ready to use the package, open it by tearing the tape.

CAUTIONS

- Do not use the indicator to monitor sterilization processes other than steam.
- Do not reuse the tape. It is recommended to remove the tape after the package has been opened and used.

STORAGE, SHELF LIFE AND DISPOSAL

- Store at normal room temperature: 15 to 30 °C (59 to 86 °F) and relative humidity of 40 to 60%. Store away from direct sunlight. Do not store near strong acidic or alkaline products, such as disinfectants or cleaning agents.
- 2 years from the date marked on the package.

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