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37414/3720 Made in Germany www.kettenbach.com

Application Instructions

After trying-in the restoration – according to the specifications in the instructions for use of Visalys® CemCore – the shade effect of the selected shade can be simulated with the compatible Visalys® CemCore Try In Paste.

In order that the natural shade effect of the core is preserved, the core surface should not be dried.

1. Selection of the required shade and application of the try-in paste with the application tip attached on the inside of the restoration. Dispose of used application tip and replace the sealing cap of the Visalys® CemCore Try In Paste.
2. Placement of the restoration on the core, the cavity or the intended tooth surface for veneers. Do not apply high pressure during placement to avoid possible breakage of the restoration.
3. Remove excess Visalys® CemCore Try In Paste using a suitable instrument (e.g. brush, cotton pellet). The best simulation of the shade effect is attained if the layer thickness of the try-in paste corresponds to the subsequent thickness of the cement layer.
4. Assessment of the shade effect. Select another shade, as appropriate.
5. Loosen the restoration and clean the adhesive surfaces of the core and the restoration with water. Attention has to be paid to complete removal of the try-in-paste to achieve a good adhesive bond for the permanent cementation.
6. Cementation of the restoration with Visalys® CemCore in the selected shade according to the instructions for use of the product.

Cleaning and Disinfection

The risk of contamination of the Visalys® CemCore Try In Paste can be reduced by covering the syringe with a disposable protective sleeve.

Disinfection of the syringe in accordance with national guidelines. Observe standard practice hygiene measures.

Storage Conditions

Store dry, protected from sunlight at room temperature (15°C/59°F – 25°C/77°F), avoiding extreme temperatures.

Do not use after the expiration date.

Constituents

The main components of the Visalys® CemCore Try In Paste are polyethylene glycol and silicic acid.

Disposal

The empty syringes and applicator tips can be disposed in the household rubbish.

Trademarks

Visalys® CemCore Try In Paste and Visalys® CemCore are registered trademarks of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials are available in certain markets only.

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal basis asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use of the product.

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