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Product Description

Mucopren® Soft is a permanently soft durable relining material for removable dentures based on vinyl polysiloxane.

Indications / Techniques

Mucopren® Soft can be applied directly or indirectly

- for permanently soft linings
- for compensating for areas of mucous membrane with varying degrees of resiliency
- for relieving sharp bony ridges
- for relieving the operation site after placing implants
- for relining temporary restorations
- for conditioning the mucous membranes in cases with denture sores
- for fabricating and relining obturators

Patient Target Group

Patients being treated in the context of a dental procedure.

Warnings

Do not use Mucopren® Soft relining material as an impression material.

Do not use Mucopren® Soft products with other reliners, sealants or adhesives.

Potential Allergenic Substances

Platinum compound, 4-methoxyphenol.

Cautions

Do not use after expiration date.

Normally, impression materials do not cause allergic reactions; however, an allergic reaction cannot be ruled out in sensitive individuals.

If in doubt, we recommend performing an allergy test before use.

Do not use the material in case of known allergies to any of the ingredients or in case of contact allergies.

Also, if used on patients whose immune system is impaired, or on those who are suffering from a variety of other diseases (such as diabetes mellitus) or those who are taking medications including antibiotics or cortisone, it has been observed

in rare cases that micro organisms (e.g. candida albicans) may colonize the soft lining. In such cases, whenever the lining of the dentures is replaced, soft lining should not be used since the recolonization by micro organisms is likely if the patient status remains unchanged.

Increased colonization by microorganisms and the development of denture stomatitis have been observed in patients due to inadequate oral hygiene and improper cleaning of soft-lined dentures.

Please instruct the patient about adequate oral hygiene and provide him with the care instructions from the base set.

Handling instructions for relined dentures are in this sheet.

Do not leave remains of soft liner in the mouth.

Do not swallow this material! If swallowed: consult a medical doctor if problems arise or persist.

Avoid contact with eyes. If accidental contact with the eyes occurs, rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Solutions used prior to the application of Mucopren® Soft (e.g., retraction solutions, mouth rinsing solutions) may interfere with the setting reaction of the relining material and must be removed by thorough spraying and drying.

If gloves are worn when processing the material, they should first be tested for compatibility using a sample. Some types of gloves can disturb the activity of the catalyst.

Should the silicone soft liner require adjustment, relieve the area liberally and apply another coat of Mucopren® Soft. Brush one coat of adhesive onto the exposed areas of acrylic only.

When handling chemicals the usual precautions must be taken.

Do not allow the material to enter the sewer or water system to avoid environmental contamination.

Avoid contact with clothing, since the material cannot be removed by dry cleaning.

Note

To ensure optimal relinings, the temperature of the material should not deviate from 23°C/74°F before applying. Otherwise, working and setting times will be affected.

Please Note – Mucopren® Adhesive

Close the bottle of Mucopren® Adhesive immediately after use.

Should the contents of the bottle be spilled, it is advisable to ventilate the area and absorb the spilled adhesive with universal binding agent.

The adhesive contains ethyl acetate / methyl methacrylate and xylene.

Instructions on the Use of Cartridge Material in the Applyfix 4 Dispensing Gun DS-50 1:1/2:1

Insert the cartridge into the Applyfix 4 dispensing gun DS-50 1:1/2:1.

Ensure that the notches on the cartridge base are pointing downwards. The clasp will not close if the cartridge is not properly inserted into the dispensing gun.

The cartridge's tamper-evident closure can now be turned (see Fig. 1).

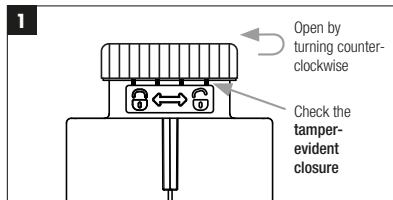
Abb. 1: Opening the tamper-evident closure

The tamper-evident closure ensures that this is an intact original product from Kettenbach Dental.

When used for the first time, the cartridge closure is visibly and audibly separated from the cartridge body. After use, the closure cap can be used to close the cartridge.

Insert the mixing tip with the integrated positioning aid into the coding hole provided for that purpose and rotate clockwise in to the thread (see Fig. 2).

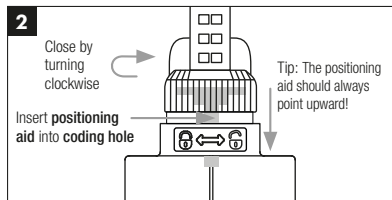
Abb. 2: Inserting the mixing tip

The intuitive handling significantly reduces mistakes when inserting.

Push the plunger into the cartridge and discard a small quantity of the relining material. Using the dispensing gun, immediately apply Mucopren® Soft evenly onto the denture base.

Replace the closure cap after use. Before using the cartridge again remove the closure cap, attach a new mixing tip and proceed as described.

Cleaning and Disinfection of the Applyfix 4

Remove the cartridge.

Remove any uncured paste using a cloth soaked in alcohol.

The Applyfix 4 dispensing gun can be used several times if properly disinfected. Kettenbach recommends exchanging the gun if there are visible traces of use. Disinfection can be carried out using a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions.

Instructions for Use for the Direct and Indirect Methods

Mucopren® Soft was developed specifically for the use of cold and hot polymerizing denture base plastics. Wash the denture base thoroughly with water and dry it. Use a bur to reduce the surface to allow for the thickness of soft liner required. To create a neat vestibular juncture between the soft liner and denture acrylic, it is advisable to trim a right-angled periphery as shown in Figure 1, page 102.

Brush a thin coating of Mucopren® Adhesive onto the prepared and roughened surface and allow to air-dry for at least 90

seconds. Ensure that the adhesive is applied **beyond the prepared periphery**.

Apply Mucopren® Soft evenly using the dispensing gun (to prevent air being entrapped, keep the opening of the mixing tip in the material) and immediately apply to the denture base. The active and passive functional excursions can then be carried out. Processing: After removing from the patient's mouth, leave the denture and the relining at room temperature for three minutes before continuing to process. A water bath is no longer needed.

Dentures which are not made of standard self-curing acrylic should be immersed in lukewarm water (30°C/86°F–35°C/95°F) for 2 hours.

Once the material has cured, cut the excess material off with scissors, a scalpel or rotary instrument to create a neat juncture between the denture acrylic and softliner (refer to Figure 2, page 102). Cross-cut steel burs for soft linings (e.g. Figur 82060 from Busch) are useful for bulk reduction. Final finishing can be carried out with the polishing discs (supplied with the basic set). Trimmed areas of acrylic can be polished as usual. Do not use polishing paste that leaves residual grease. Always clean the surfaces with hot water (steam cleaner) and dry them before sealing.

Lastly, apply a thin layer of Mucopren® silicone sealant to the prepared area of soft liner. It is advisable to place a small

amount of Mucopren® silicone sealant into a suitable vessel (e.g. the plastic one supplied in the pack) and then apply it with a brush within 90 seconds (see Figure 3, page 103). The silicone loses its adhesive properties after 6 minutes and 30 seconds. Please note: The silicone sealant does not attain its final mechanical properties until it has been at mouth temperature for several hours.

When relining indirectly, apply adequate amounts of separating agent to the model (use a separator which forms a micro-film e.g. Dentsply Isolant®). As agar-agar based separating agents may form a culture medium for micro organisms such as candida albicans, they must not be used. Place the reline jig or flask in a pressure vessel or a water bath containing water at $\geq 50^{\circ}\text{C}$ / 122°F for at least 3 minutes.

Storage Conditions

Store in a dry place, protected from sunlight and at room temperature ($10^{\circ}\text{C}/50^{\circ}\text{F} - 25^{\circ}\text{C}/77^{\circ}\text{F}$); avoid extreme temperatures. Do not use after the specified expiration date.

Disinfection

Soft lined dentures can be disinfected (immersion disinfection).

The cross-cut steel bur for permanent soft relining materials (Figur 82060 from Busch) that is included in the basic set is manufactured from normal steel and is therefore not suitable for reprocessing in a steam steriliser.

It is also important for normal steel to ensure that disinfectants and cleaning agents are used with corrosion protection.

Composition

Vinyl polysiloxanes, Hydrogen polysiloxanes, Filler, Pigments, Additives, Platinum compound.

Disposal

The empty cartridges/mixing tips/applicator tips and set material can be disposed in the household rubbish.

Trademarks

Mucopren® is a registered trademark of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials/products are available in certain markets only.

For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority (for Germany: the Federal Institute for Drugs and Medical Devices, BfArM).

Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

Updated: August 31, 2023

Product Overview

Product name	ISO 10139-2	Mixing ratio and total content volume per unit	Mixing technique	Mixing element
Mucopren® Soft	Type A	1:1, 50 ml cartridge	Applyfix 4 dispensing gun DS-50 1:1/2:1	mixing tip, blue, Ø 6.0 mm
Mucopren® silicone sealant	—	1:1, 50 ml cartridge	Applyfix 4 dispensing gun DS-50 1:1/2:1	mixing tip, red, Ø 4.0 mm

Technical Data

Product name	Mixing ratio	Working time at 23°C/ 74°F ≤	Working time at 35°C/ 95°F ≤	Intraoral setting time at 35°C/ 95°F ≥	Total setting time ≥	Hardness (approx.) Shore	Linear dimensional change (approx.) %	Elastic-Recovery test (approx.) %	Strain in compression (approx.) %
Mucopren® Soft Direct reline	1:1	2 minutes 15 seconds	not applicable	3 minutes 15 seconds	5 minutes 30 seconds*	A 28	– 0.20	99.5	9.4
Mucopren® Soft Indirect reline	1:1	2 minutes 15 seconds	not applicable	not applicable	> 3 minutes in the pressure vessel or water bath at ≥ 50°C/ 122°F	A 28	– 0.20	99.5	9.4
Mucopren® silicone sealant	1:1	1 minute 30 seconds	not applicable	not applicable	6 minutes 30 seconds at 23°C/74°F	A 33	– 0.20	—	—

* Total setting time (removal time from mouth) from start of mix

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DEAR PATIENT,

English

The bottom of your denture has been fitted with a permanently soft lining to make it as comfortable as possible for you to wear. The soft lining does not smell or taste at all.

Please do not eat for the first two hours or clean your denture by hand, e.g. with a toothbrush, for the first day. In individual cases, the consumption of coffee, carrot juice and red wine might cause discolorations of the denture.

You can help make the soft lining last longer by caring for it carefully. Rinse your denture daily in running water before cleaning it with any established brand of denture cleanser based on active oxygen (e.g. Blend-a-dent 2-Phase, Corega Tabs or Kukident). This reduces colonization by microorganisms. Then carefully clean your denture with a soft toothbrush and your usual toothpaste. Substances that do contain alcohol are not suitable for cleaning your dentures as they can have an adverse effect on the permanently soft lining.

Once you have cleaned your denture, rinse your mouth thoroughly and replace the denture in your mouth. Alternatively, keep your denture in water or a dry place – depending on what your dentist advises. In any case, follow your dentist's advice.

Should parts of the soft lining become detached due to overloading, your dentist can replace them.

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