Adhesive for Kettenbach

addition-curing silicones

PANASIL® HAFTLACK

INSTRUCTIONS FOR USE / Gebrauchsinformation / Mode d'emploi / Gebruiksaanwijzing / Инструкция по использованию / Istruzioni per l'uso / Instrucciones de uso / Kullanım Kılavuzu / Instruções de uso / Instrukcja użycia / Инструкция для застосування / 取扱説明書 / تعليمات الاستخدام

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ENGLISH

Product Description / Indication

Adhesive for Kettenbach addition-curing impression and reline impression materials. Adheres to metals and plastics.

Instructions for Use

The surface of the impression tray must be repeatedly and fully brushed so that the entire surface of the tray is visibly blue. Allow to dry for about 5 minutes. Warmth and compressed air shorten the drying time. For impression trays made of plastics, please check the compatibility of the adhesive. Close the bottle immediately after use!

After mixing the impression material, load it into the tray within 30 minutes.

Remains of impression material can be cleaned off the tray with cleaning solvent, petroleum ether or orange oil.

Warnings

Do not use Panasil® Adhesive with condensation-curing silicones, polyether or polysulfide materials.

Precautions

Do not use after the expiration date.

Do not swallow the material! If accidentally swallowed: consult a medical doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Avoid contact with skin.

Do not allow the material to enter the sewer or water system to avoid environmental hazard.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Keep away from sources of ignition.

Side Effects

Although there are no known systemic side effects, specific contact allergies cannot be ruled out in rare cases. Do not use the material in case of known allergy to one of the ingredients.

Composition

Silicone resin, solvent (ethyl acetate, xylene)

Storage Conditions

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

Disposal

The empty bottles can be disposed in the household rubbish.

Trademarks

Panasil® is registered trademark of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials are available in certain markets only.

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

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Manufacturer: Kettenbach GmbH & Co. KG