EXAlence™ 370VINYL POLYETHER SILICONE (VPES™) IMPRESSION MATERIAL
Für use only by a dental professional in the recommended indications.

Viscosity	Setting	Type of Mixing Tip
MEDIUM BODY (MONOPHASE)	Regular & Fast Set	Dynamic Mixing Tip (yellow)
HEAVY BODY	Regular & Fast Set	Dynamic Mixing Tip (yellow)

RECOMMENDED INDICATIONS and Impression techniques

Indication	Impression Technique	Tray Type	Recommended Viscosities
	Single mix single impression		Medium Body
Impression for facetas, inlays, onlays, crown or bridge	Double mix single impression	Stock or Custom Tray	Heavy Body or Medium Body + (Extra) Light Body
	Double mix double impression (Putty-wash impression)		Putty or Heavy Body + Medium Body or (Extra) Light Body
	Triple tray technique	Triple Function Tray	Heavy Body (Rigid) + (Extra) Light Body
Impression for partial denture	Single mix single impression	Custom Tray	Medium Body
Impression for full denture	Single mix single impression	Custom Tray	Medium Body or Light Body

CONTRAINDICATIONS

Avoid use of this product in patients with silicon silicons and flavoring allergies.

PHYSICAL PROPERTIES (Average)

Test conditions: Temperature (74°F/23°C ± 4°F/2°C) Relative humidity (50 ± 5%)

	Medium Body (Monophase) (Type 2: Medium Consistency)	Heavy Body (Type 1: High Consistency)
Total working time (minutes)	Regular: 2 min. Fast: 1 min.	Regular: 2 min. Fast: 1 min.
Minimum time in mouth (minutes)	Regular: 3 min. Fast: 1.5 min.	Regular: 3 min. Fast: 1.5 min.
Recovery from Deformation (%)	Regular: 98.4% Fast: 97.0%	Regular: 98.1% Fast: 98.1%
Maximum strain in compression (%)	Regular: 2.9% Fast: 3.4%	Regular: 3.0% Fast: 3.2%
Linear dimensional change (%)	± - .10 %	± - .10 %
Detail reproduction	20µm	50µm
Compatibility with gypsum	20µm	50µm

PRECAUTIONS

When extracting EXAlence, care should be taken to avoid mixing or contact with the following materials. These may delay or prevent setting; catalyst for condensation silicone impression materials; polysulfide impression materials; eugenol materials, sulfur, latex, oil, acrylates, glycerol, spray-type or oil-remint-type topical surface anesthesia (e.g. lidocaine).

DIRECTIONS FOR USE**CARTRIDGE LOADING AND DISPENSING**

NOTE: The cartridges are heavy and dropping may result in damage to the cartridges rendering it unusable. The cartridge and mixing tip provided are compatible with a dynamic mixer. Also refer to the manufacturer's instructions.

- Hold and support the cartridge with the extrusion nozzle's facing upwards.
- Grasp the tab on the cap and bend in an upward 90° direction. Do not place excessive stress on the tab.
- While holding the cartridge securely, grasp the tab with thumb and forefinger and remove the cap from the cartridge.
- Insert the cartridge into the mixer according to the manufacturer's instructions.
- Start the mixer according to the manufacturer's instructions. Blend a small amount of material from the two nozzles of cartridge prior to each use. Wipe off the extruded material in a vertical direction to avoid cross contamination of the nozzle.
- Install a supplied mixing tip on the extrusion nozzle following the manufacturer's instructions. **NOTE:** Do not use other mixing tips. They will not fit properly and can cause material to leak from the nozzle.
- If the mixing tip does not seat easily, be sure the internal central hex of the mixing tip is properly aligned with the drive shaft of the mixer.
- Once the mixing tip is properly placed, place nozzle locking ring over the mixing tip and turn it 1/4 turn clockwise until ring is locked in place.

IMPRESSION STEP

- NOTE: Prior to use, leave the material to stand at room temperature. Cooled or refrigerated material may delay setting.
- Make sure the preparation is dry and free of debris during impression taking.
 - Block out undercuts to aid impression removal.
 - Apply a tray adhesive (GC Universal VPS Adhesive, etc.) onto the tray and leave to dry. Read manufacturer's instructions for use.
 - Press the button and dispense the mixed base and catalyst into an intraoral syringe (Monophase type only) or suitable tray.
 - Immediately after dispensing, move the plungers of the machine off the cartridge to prevent continued pressure on the material and extrusion of excess material from the tip.
 - Inject the wash material onto the prepared teeth and completely cover the preparation. Blasting with air is not necessary. EXAlence's hydrophilic property enables better self-wetting, allowing an excellent adaptation to the tooth.

- Note:** In case of using the double mix double impression or Putty-wash technique, make sure the preliminary impression is cleaned and thoroughly dried prior to taking the final impression. Set the loaded tray in the mouth. See above table for recommended working times.
- Wait for the material to set. See above table for recommended intra oral setting times.
 - EXAlence has unique chemistry that allows it to polymerize faster in the oral cavity than at room temperature. Therefore to determine setting, examine material in the mouth using a material on bench will yield a false result. Material at room temperature will not set within acceptable time.
 - Higher temperatures shorten working time and lower temperatures increase working time.
 - Remove impression and rinse under running tap water.

DISINFECTION

The impression can be disinfected with a glutaraldehyde solution or other appropriate disinfectants. Refer to the respective manufacturer's instructions for use.

MODEL FABRICATION

- After the impression is rinsed with water, wait for 60 minutes before pouring a model.
- Pour a model within 14 days after impression taking. EXAlence is compatible with type III and type IV stone (GC Fuji ROCK™ EP) and can be silver or copper plated.

CAUTION

- Avoid contact of base, catalyst, mixed material or adhesive with eyes. In case of contact, immediately flush with generous amount of water and seek medical attention.
- Avoid getting material on clothing as it is difficult to remove.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

STORAGE

Recommended for optimal performance, store at room temperature (59° - 77°F; 15° - 25°C). Avoid direct sunlight.

PACKAGES

- | | |
|--|---|
| 1. EXAlence 370 MEDIUM BODY (MONOPHASE)
Regular and Fast Set
511g (370ml) x 1 cartridge
Mixing tip for 370ml Cartridge (yellow) x 10 pieces | 4. EXAlence 370 MEDIUM BODY (MONOPHASE) REFILL
Regular and Fast Set
511g (370ml) x 2 cartridges
1 Locking Ring (U.S./Canada only) |
| 1 Locking ring
584g (370ml) x 1 cartridge
Mixing tip for 370ml Cartridge (yellow) x 10 pieces
1 Locking ring | 5. EXAlence 370 HEAVY BODY
Clinic Pack
Regular and Fast Set
584g (370ml) x 6 cartridges
1 Locking ring (U.S./Canada only) |
| 3. EXAlence 370 HEAVY BODY REFILL
Regular and Fast Set
584g (370ml) x 2 cartridges
1 Locking Ring (U.S./Canada only) | 6. EXAlence 370 MEDIUM BODY (Monophase) Clinic Pack
Regular and Fast Set
511g (370ml) x 6 cartridges
1 Locking Ring (U.S./Canada only) |
| | 7. Dynamic Mixing Tip (Yellow)
50 pieces |
| | 8. Bayonet Locking Ring 5 pieces |
| | 9. Universal VPS Adhesive
1 bottle (7mL) |

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gcorp.com> or for The Americas: <http://www.gametrica.com>. They can also be obtained from your supplier.

CLEANING AND DISINFECTING
MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERGE: Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

UNDESIRED EFFECTS - REPORTING:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

Last revised: 01/2020

EXAlence™ 370VINYL POLYETHER SILIKON (VPES™) ABFORMMATERIAL
Für use only by a dental professional in the recommended indications to verwenden

Viskosität	Abbindeform	Mixing Tip Typ
MEDIUM BODY (MONOPHASE)	Regular & Fast Set	Dynamischer Mixing Tip (gelb)
HEAVY BODY	Regular & Fast Set	Dynamischer Mixing Tip (gelb)

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