PUMA 3000

MANUAL

- en** > BRUSHLESS ELECTRICAL MICROMOTOR FOR LABORATORY & CONTROL BOX
- es** > MICROMOTOR ELÉCTRICO PARA LABORATORIO SIN ESCOBILLAS Y CAJA DE CONTROL
- it** > MICROMOTORE ELETTRICO DA LABORATORIO SENZA SPAZZOLE E SCATOLA DI CONTROLLO
- fr** > MICROMOTEUR ÉLECTRIQUE SANS BALAIS POUR LABORATOIRE ET BOÎTE DE CONTRÔLE
- pt** > LABORATÓRIO BRUSHLESS MICROMOTOR ELÉCTRICO E CAIXA DE CONTROLO
- de** > ZAHNHEILKUNDELABORE BÜRSTENLOSER ELEKTRISCHER MIKROMOTOR

NEXO

Enjoy dentistry.

- en** > Please read this manual before use and file future reference.
- es** > Lea este manual antes de utilizarlo y archívelo para futuras consultas.
- it** > Leggere il presente manuale prima dell'uso e come riferimento futuro.
- fr** > Veuillez lire ce manuel avant de l'utiliser et le classer comme référence future.
- pt** > Por favor leia este manual antes de utilizar e archive referências futuras.
- de** > Bitte lesen Sie dieses Handbuch vor dem Gebrauch und zum späteren Nachschlagen.

English (Pág 4-8)

1. **FEATURES OF PUMA3000**
2. **TECHNICAL SPECIFICATIONS**
3. **INSTRUCTIONS**
4. **WARRANTY**
5. **TRANSPORT AND STORAGE**
6. **CONTENTS**
7. **DIAGRAM**

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INSTRUCTIONS FOR USE

The PUMA3000 micromotor belongs to the brushless permanent magnet micromotors for use by the dental technician, also suitable for dental clinics and all types of factories.

1 > FEATURES OF PUMA3000

- ISO standard connector, can be used with all straight and contra-angle E-type heads.
- PC board power supply input: 22~24 V AC/DC 50/60Hz
- Control panel has (forward/reverse) switch, continuous speed control (hand/foot), digital display showing speed.
- The foot panel should be connected to its socket on the back of the control box.

2 > TECHNICAL SPECIFICATIONS

- Input voltage: 100-240VAC 50/60Hz
- Power: 30W
- Rotation speed: Min 2,000rpm/M---- Max 40,000rpm/M
- Interface: ISO-E type
- Way of control: Hand or foot control

- Torque: 150gf.cm

3 > INSTRUCTIONS

Warning: Please make sure that the power switch is turned off (if not, the unit may be damaged).

1. Check that the power switch is turned off.
 - a. Connect the cable.
 - b. Connect the foot pedal to its connector.
 - c. Connect the handpiece to its connector.
2. Turn the speed regulator of the control box to "0".
3. Check that the clamp is securely locked by turning the handpiece handle or pulling the tip. Hold the handpiece with one hand and turn on the power switch with the other hand.
4. Turn on/off: press the power switch, the green light of the switch will turn on and the display will show the number of rpm. Or it will turn off.
5. Choose between manual or foot pedal control with the hand/foot switch. Each press switches between manual and foot pedal.
6. Choose the direction of rotation with the right/left selector. Each press changes the direction of rotation between right and left.
7. Use the MOTOR ON/OFF button to run or stop the motor both in manual mode and via the foot pedal.

8. Change speed, depending on the control mode you have chosen hand/foot. Turn the speed controller clockwise or press the foot pedal.

NOTE

- The PUMA3000 micromotor is a maintenance-free, shock-protected and lubrication-free product.
- Please make sure to use the correct voltage when plugging in.
- Switch off and unplug it when not in use.
- For use by qualified persons only.
- It is forbidden to connect or remove the handpiece when the motor is running.
- Please make sure the handpiece is securely clamped before use.
- The foot pedal connector should be inserted into the rear socket driver, do not connect to the voltage connector (110V or 220V).
- If any problem occurs during use, please unplug and discontinue use immediately and notify the manufacturer or authorized distributors.
- Repair and maintenance should only be performed by qualified personnel.
- Please operate with care to avoid injury.

4 > WARRANTY

The PUMA3000 motor is warranted for 1 year for its handpiece, control box and foot pedal.

The warranty does not include failures due to misuse or normal wear of bearings, collet, spindle and armature. In case of incorrect power supply voltage. In case of using spare parts or accessories not recommended by the manufacturer. - In case of repairs not carried out by the manufacturer.

5 > TRANSPORT AND STORAGE

- Disassemble carefully, protect from shocks and humidity.
- Store in a dry and ventilated place.

6 > CONTENTS

- Control box, 1 piece
- Handpiece, 1 piece
- Handpiece holder, 1 piece
- Foot control, 1 piece
- Fuse, 1 piece

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Being Foshan Medical Equipment Co., Ltd
Shanghai International Holding Corp. GmbH