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Description:

Opalescence Go is a dentist-controlled, dentist-supervised, take-home whitening system dispensed in pre-loaded disposable trays. It is a clear, flavored, high viscosity, sticky, 6% hydrogen peroxide gel with potassium nitrate and sodium fluoride.

Precautions for Europe:

1. Not to be used on a person under 18 years of age.
2. To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.
3. Contains sodium fluoride.
4. Please note the expiration date printed on the outer package/tray containers.
5. Some patients may experience temporary discomfort of the gums, lips, throat, tongue, or teeth. Should any of these symptoms persist or worsen after discontinuing use, or in cases of severe discomfort, call your dental professional.
6. Do not eat, drink, or use tobacco products while whitening trays are in place.
7. Do not wear whitening tray while sleeping.
8. Avoid swallowing large amounts of gel. Ingestion of small amounts of gel during use is expected.
9. For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.
10. Restorative materials will not whiten.
11. Inform your dentist that you have been whitening prior to any dental procedure.
12. Patients with serious health concerns should consult their primary care provider prior to treatment.
13. Pregnant or nursing women should not use teeth whitening products.
14. White spots on the enamel may become more apparent during the whitening process and often blend with continued whitening.
15. Teeth can re-stain over time, do touch up teeth whitening treatments as necessary.
16. Avoid contact with eyes.
17. To prevent additional irritation, gingival and general oral health should be evaluated prior to initiating treatment.
18. Monitor progress to prevent whitening beyond the desired shade or translucency.

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