· Conscious dentistry ·

VIDU

CHROMATIC ALGINATE

ES · ALGINATO CROMÁTICO
IT · ALGINATO CROMATICO
FR · ALGINATE CHROMATIQUE
PT · ALGINATO CROMÁTICO
DE · CHROMATISCHES ALGINAT

“THE FUTURE
IS IN YOUR HANDS”

VIDU
CHROMATIC ALGINATE

EN

INSTRUCTIONS FOR USE

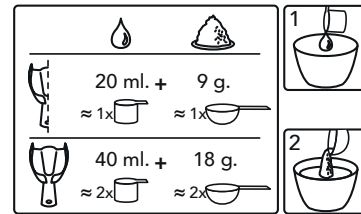
CHROMATIC ALGINATE VIDU is an alginate fast set, type 1, does not require special treatment before use. Dustfree material. However, caution is advised when opening the bag and taking the powder. While tap water may be used, the declared times were measured by mixing the powder with deionized water at 23°C. **Vigorously mix the powder with water.** When using specific mechanical mixers for alginate-based impression materials, follow the manufacturer's instructions.

Follow the chromatic phase indications which ensure that the product is mixed perfectly and inserted into mouth at the right moment.


Variations in temperature and/or water hardness and the speed and/or friction during mixing cause small changes in the working time and in the initial setting time, which are automatically indicated by the color changes.

Wash, rinse and shake the impression and pour the model. The model can also be poured within up to 1120 hours with excellent results. The impression must be stored in a sealed plastic bag without adding anything else. **Do not leave the impression exposed to air or immersed in water.**

Keep the container tightly closed in a cool, dry place. Excellent results can be obtained with VIDU plasters.

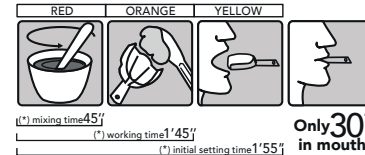(*) Measurements taken at 23°C with deionised water.

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
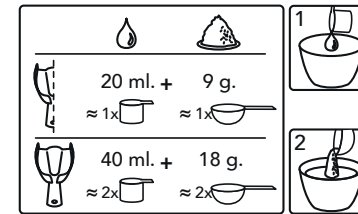
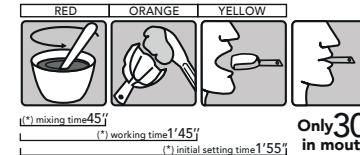
(*) Measurements taken at 23°C with deionised water.

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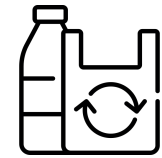
- [Ref. VI41003] ALGINATO CROMATICO VIDU, 450 gr.

ECO
DESIGN

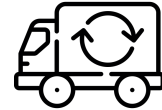CLOSE
MANUFACTURE

RESPONSIBLE
CONSUMPTION

SMART
WASTE

ONLINE
DISTRIBUTION

Dentared Odontology Services SL
C. Torres Quevedo 13 Alcobendas,
28108, Madrid, Spain
www.vidu.eco

LASCOD S.P.A.
Via L. Longo 18,
50019 Sesto Fiorentina,
Florence, Italy

MEDICAL DEVICE
DUST FREE - ISO 21563
MADE IN ITALY

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