INSTRUCTIONS FOR USE

CALCIUM HYDROXIDE PASTE LC VIDU is a light-cured resin-modified calcium silicate, calcium hydroxide pulp protectant/liner designed to perform as a barrier and to protect the dental pulpal complex. CALCIUM HYDROXIDE PASTE LC VIDU precise placement allows its use in all deep cavity preparations. The proprietary hydrophilic resin formulation creates a stable and durable liner or base.

Indications for Use

- 1) CALCIUM HYDROXIDE PASTE LC VIDU is intended for use as a direct pulp capping agent.
- 2) CALCIUM HYDROXIDE PASTE LC VIDU is intended as an indirect pulp capping agent in deep preparations, a protective liner in large preparations, base or sealer for use:

 - Under amalgam restorations
 - Under Class I and Class II composite restorations
 - Under other base materials
 - Under cements
 - As a base or liner under deciduous

Contraindications

If a dry working field cannot be established, or the stipulated working technique cannot be applied. - If a patient is known to be allergic to any of the ingredients of CALCIUM HYDROXIDE PASTE LC VIDU

Application

- 1) Under rubber dam isolation, complete cavity preparation.
- 2) Apply CALCIUM HYDROXIDE PASTE LC VIDU directly to the exposed pulp in incremental layers. Layer is not to exceed 1 mm in depth
- 3) Light cure each increment for 20 seconds.
- 4) Place desired adhesive, base, and/or restoration following manufacturer's directions

Precautions

1. CAUTION: Chemical may set in tip after a few hours. To avoid clogging, remove tip and re-cap with Luer Lock Caps between each use.
2. Carefully read and understand all instructions before using CALCIUM HYDROXIDE PASTE LC VIDU. Use only as directed.
3. For daily use, store at room temperature. Refrigerate bulk storage.
4. Redirect overhead light to prevent premature polymerization.
5. Resins can be sensitizing. Avoid repeated contact of uncured dental resin with skin. Do not use on patients with a known sensitivity to acrylics or other resins.
6. Dispose of used tips and empty syringes properly.
7. Keep out of reach of children.
8. To avoid cross contamination, do not re-use tips.
9. Do not use after expiration date noted on containers.
10. To avoid cross contamination, disinfect and wipe syringe with an intermediate level disinfectant between uses and/or use a disposable syringe cover. If syringe cover is used, remove tip, re-cap, and discard tip and syringe cover.
11. Keep CALCIUM HYDROXIDE PASTE LC VIDU out of heat/direct sunlight.

Side effects

In individual cases, components of CALCIUM HYDROXIDE PASTE LC VIDU may lead to sensitization. CALCIUM HYDROXIDE PASTE LC VIDU should not be used in such cases. To avoid possible irritation of the pulp, areas close to the pulp should be protected with a suitable pulp/ dentin protector (selectively apply a calcium hydroxide-based preparation in areas close to the pulp).

Interactions

Substances containing eugenol/ clove oil may inhibit the polymerization of composite materials. Consequently, the application of such materials together with CALCIUM HYDROXIDE PASTE LC VIDU must be avoided. Cationic mouth rinses and plaque disclosing agents as well as chlorhexidine may cause discoloration

Storage

Store at temperature 1-30 OC / 33,8-86 OF
Disposal: Disposal must be made according to official regulations. Take to an approvedlandfill or a waste incineration plant, under conditions approved by the local authority.
Uncleaned packagings: Disposal must be made according to official regulations

Cautions

CALCIUM HYDROXIDE PASTE LC VIDU contains polymerizable methacrylate monomers. Avoid prolonged exposure with skin, oral soft tissues, and eyes. Unpolymerized CALCIUM HYDROXIDE PASTE LC VIDU may have a slight irritating effect and may lead to a sensitization against methacrylates. Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates. Applicator tips are single use
Use of the information for preparation date:

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