· Conscious dentistry ·

VIDU

PORCELAIN ETCHING GEL

en · Porcelain etching gel
es · Gel grabador de porcelana
it · Gel mordicante per porcellana
fr · Gel de mordantage porcelaine
pt · Gel grabador de porcelana
de · Porzellan ätzgel

"THE FUTURE IS IN YOUR HANDS"

VIDU

PORCELAIN ETCHING GEL

en

INSTRUCTIONS FOR USE

Description

5% hydrofluoric acid gel for etching ceramic restorations. It is used to roughen the cementation surface of all ceramic restorations.

Composition

Contains 5% hydrofluoric acid.

Indications

- PORCELAIN ETCHING GEL VIDU is used for pickling broken dental porcelain and porcelain veneers, crowns or inlays in the Office
- It is also used for pickling porcelain crowns before bonding orthodontic brackets.

Contraindication

There is a contraindication if the prescribed application technique cannot be applied.

Side effects

There are no known side effects to date.

Application

1. Refresh the porcelain surface to be repaired with a high speed diamond bur. If a repair has been done previously, remove all of the old composite for a fresh porcelain surface.
2. Before applying, drop a small sample of the acidifier onto the mixing pad from the tip to ensure a smooth flow and view the flow rate. DO NOT continue if there is excessive resistance to flow. Attach a new tip and retest before use on the patient. Dispose of the removed tip immediately (in accordance with official regulations) to avoid contact.
3. Apply the PORCELAIN ETCHING GEL VIDU in an even layer on the porcelain prepared by using tip for 90 seconds.
4. While the dentist is applying PORCELAIN ETCHING GEL VIDU, the assistant should hold the suction tip adjacent to the surface. Thus, while the acid reacts with the porcelain (90 seconds), the irritating steam is removed.
5. Remove neutralized acid by placing the vacuum nozzle close to the surface, and then rinsing with water. Rinse well and dry. Porcelain should have a frozen appearance. If not, repeat the pickling process.
6. Apply PORCELAIN ETCHING GEL VIDU for 5 seconds to remove porcelain compounds and residues created by the hydrofluoric etchant process. Wash and dry.
7. Apply silane using a Mini Brush tip. Wait 60 seconds for it to evaporate. If it does not dry completely after 60 seconds, dry it using oil-free air.
8. Apply bonding resin per product instructions.

Measures

1. The chemical may freeze at the tip after a few hours. To avoid clogging, remove tip and re-cap after each use. Always check whether the material is flowing before applying it into the mouth. If resistance is encountered, replace the tip and check again. This does NOT apply to dual-barrel syringes and mixing tips.
2. The assistant and the patient should wear protective glasses.
3. Attach caps to syringes when not in use. Tips are disposable
4. While using and applying this product, the physician and the patient should wear side shield protective glasses.

Warnings

1. For professional use only.
2. Review instructions, precautions and Material Safety Data Sheet (SDS) before starting treatment. Use only as directed.
3. Keep the products away from heat / sunlight.
4. Avoid contact of resins with skin.
5. Keep strong chemicals away from the treatment area.
6. Confirm that the patient has no known allergies to the treatment materials.
7. Check whether the materials are flowing from the syringe and tip before applying into the mouth.
8. Never force the syringe plungers.

Storage

Do not use Etch after the date of expiration.
Storage temperature: 1-30 °C (33,8-86 °F)
Recommendation: Refrigerate large inventory at 2-8 °C (36-46 °F).

Disposal

Disposal must be made according to official regulations. Take to an approved landfill or a waste incineration plant, under conditions approved by the local authority. Uncleaned packagings: Disposal must be made according to official regulations

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