## ABSORBENT PAPER POINTS

ES · PUNTAS DE PAPEL ABSORBENTES  
IT · PUNTE DI CARTA ASSORBENTE  
FR · POINTES DE PAPIER ABSORBANT  
PT · PONTAS DE PAPEL ABSORVENTES  
DE · SAUGENDE PAPIERSPITZEN

“THE FUTURE  
IS IN YOUR HANDS”

VIDU

EN

INSTRUCTIONS FOR USE

### 1. Brief of the product

The Sterile Absorbent Paper Points belongs to dental special root canal cleaning material, mainly used for root canal cleaning in pulpitis treatment. It is sterilized one-time use instruments.

### 2. Product Structure and materials

This product is made of high-grade ultra-thin cotton paper. It has good moisture absorption performance and no irritation to root canal tissue.

### 3. Scope of application

This product is a special dental root canal cleaning material, mainly used for root canal cleaning in the treatment of pulpitis.

Warnings

- 3.1 Only skilled dentists are allowed to use.
  - 3.2 Be sure to sterilize this product for each use.
  - 3.3 Do not use this product except for the dental service and treatment. Use it in accordance with the intended use.
  - 3.4 Paper Points cannot be cleaned or sterilized. Cleaning or sterilization may cause the device to become unusable.
- Contraindications and prohibitions  
Don't use this product for patient who indicates sensitization and allergic reaction.

### 4. Re-Processing

The instruments are single use and not approved for re-use.

### 5. Specifications and Method of Use

5.1 The Specification of VIDU Absorbent Paper Points as the below

- 1) ISO standard type  
15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80, 90, 100, 110, 120, 130, 140
- 2) Great Taper Type:  
- Taper ISO .04: 15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80, 90, 100, 110, 120, 130, 140  
- Taper ISO .06: 15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80, 90, 100, 110, 120, 130, 140
- 3) Accessory Type: XXF, XF, F, M, C, XC

5.2 Method of Use:

After uncovering the pulp of the bad tooth and exposing the root canal orifice on the top of the tooth, use tweezers to clamp the stained part of the absorbent paper tip with the same model as the enlarged needle to clean the cavity, and suck out the tissue fluid, pus, blood and other fluids in the root canal. The following steps summarize: 1) Select the paper point that best matches the last instrument used to shape the canal. 2) Dry the canal by gently inserting paper points to the final working length (apex).

### 6. Disinfection and Sterilization before use

1) Ensure the canal is completely dried following use of Paper Points. Failure to properly dry the canal could lead to patient sensitivity or procedural delays. 2) Do not apply excessive force during lateral condensation to prevent vertical fracture of root. 3) Ensure proper site preparation to remove infected tissue/bone/dentin to prevent a biomaterial centered infection (BCI). 4) Follow proper aseptic techniques to prevent contamination, which could result in a BCI. 5) Ensure the Paper Points isn't accidentally introduced into the maxillary sinus as this could cause foreign body inflammation or maxillary sinus Aspergillosis. 6) Follow proper obturation techniques to avoid procedural accidents, which could result in persistent apical periodontitis infection. 7) Ensure removal of the Paper Point or fibers from the root canal after use. Failure to do so can cause post endodontic periapical inflammation, exuberant granulation of tissue or periapical cysts. 8) Do not use distorted or flimsy Paper Points as this could result in poor drying of the canal or procedural delays. 9) Always check the expiration date of the product to prevent procedural delays or user inconvenience (e.g. Paper Points unroll). 10) Carefully read package labeling to ensure use of the appropriate Paper Points. Failure to do so may cause procedural delays.

### 7. Sterilization

The product is completely packaged, it is sterilized by y-ray cobalt 60 irradiation, and it is regarded as a sterile product. When the packaging is damaged, it cannot be regarded as a sterile product.

### 8. Storage

Keep devices in sterilization packaging in a clean environment, away from sources of moisture and direct sunlight. Store at ambient temperature. Stored in a well-ventilated room with a relative humidity of no more than 80%, no corrosive gas. Guarantee sterility cant' if packaging is open, damaged or wet. Check the packaging and the medical devices before using them (packaging integrity, no humidity and use by date).

### 9. Duration of use

The shelf time is 3 years.

### 10. Disposal

For proper disposal, always observe national laws and recommendations of the authorities.

### 11. Packaging

Packaging unit: 200pcs/package or 100pcs in a plastic box.

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