



SURGISPON[®] DENTAL

Absorbable Haemostatic
Gelatin Sponge



Fast &
effective



Absorbs approximately
45 times of its own weight



Saves valuable surgical
& anaesthetic time

SURGISPON®

SURGISPON® is a sterile haemostatic absorbable gelatin sponge made from highly purified pharmaceutical grade gelatin (lyophilized hydrolyzed collagen) foam, which gives it a uniform porosity and reacts neutrally. The dried foam is cut, packaged and sterilised using gamma radiation.

HOW to use SURGISPON® Dental

SURGISPON® may be used in combination with antibiotics, thrombin, chemotherapeutics without reduction of the sponges haemostatic effect. Carefully read applicable instructions for use (IFU) before application.

Products



SURGISPON® is immediately available for use in the operating theatre and does not require any sterilisation by dry heat or autoclaving.



WHERE to use SURGISPON® DENTAL

To achieve immediate haemostasis in
Extractions, Oral Bone & Soft Tissue Surgery.



Tooth Extractions

Oozing and bleeding from alveolus is stopped by application of SURGISPON® Dental cube.



Enucleation of Cysts

Bleeding from the cavity is controlled with the Dental cube.



Detail

Dental cube placed in cavity.



Control of Profuse Bleeding

In tooth extraction by means of several Dental cubes.

Ordering Information



SSP-101010
CUBES



SSP-1208
DENTAL TAMPON



SSP-202007
SHEET

Product ID	Product Name	Dimensions (mm)	No. of Pieces Per Box
SSP-101010.12	CUBES	10X10X10	12
SSP-101010.32	CUBES	10X10X10	32
SSP-101010.50	CUBES	10X10X10	50 per jar
SSP-1208	DENTAL TAMPON	12X08 Ø	32
SSP-202007	SHEET	20X20X07	20



Aegis Lifesciences Private Limited

215/216, Mahagujarat Industrial Estate,
Ahmedabad, Gujarat-382213, India.

☎ +91 9512503131 📞 +91 9512603131

✉ marketing1@aegis-lifesciences.com
aegis.lifesciences@gmail.com



BROCHURE



WEBSITE



YOUTUBE