UltraTemp® And UltraTemp® REZ Temporary Cement
--

Description
UltraTemp® is a non-eugenol temporary cement. It is polycarboxylate based and does not negatively affect resin bonding.
UltraTemp® REZ is a non-eugenol temporary cement. It is resin based and does not negatively affect resin bonding.
Indications for Use
For temporary application of provisional crowns, bridges, inlays, and onlays.

Warning: Peroxides will negatively affect the set of UltraTemp REZ.
Directions for Use

- Remove cap by twisting counterclockwise and discard.
- Prior to attaching mixing tip, verify uniform flow from both syringe orifices by expressing a small amount onto a mixing pad. Use caution not to cross-contaminate orifices.
- Attach mixing tip by lining up internal cap stems with syringe orifices. Twist clockwise to lock.
- Clean, rinse, and dry preparation. If tooth is sensitive, patient may require anesthesia.
- If using intraorally, firmly attach an Intraoral tip.
- Express desired amount of UltraTemp® or UltraTemp® REZ into syringe or access opening.
- Seat provisional. Remove excess unset cement using a wet cotton pellet.
- After complete set, remove all excess cement with a hand instrument.
- Disinfect and wipe mixing tip and syringe. Do not remove mixing tip until next use.

	Set Time
UltraTemp®	2 - 3 minutes
UltraTemp® REZ – Regular Set	2 - 3 minutes
UltraTemp® REZ – Fast Set	1 - 2 minutes

Removal of UltraTemp® / UltraTemp® REZ from tooth surface

- Remove provisional.
- Flake off residual cement with blunt hand instrument.
- Scour preparation with pumice-type slurry and cup or brush or ConsepSis® Scrub and the Ultradent STARbrush®.
- Rinse thoroughly prior to placing definitive restoration.

Precautions and Warnings

- Carefully read and understand all instructions before using.
- Keep out of reach of children.
- Do not use UltraTemp® REZ with peroxide based chemicals (i.e. walking bleach).
- Do not use on patients with known sensitivities to polyacrylic acid (polycarboxylate).
- Resins can be sensitizing. Avoid repeated contact of uncured dental resin with skin. Do not use on patients with a known sensitivity to acrylates or other resins. If allergic reaction, dermatitis or rash develops, consult a physician.
- Avoid cross-contamination of syringe orifices which could result from repeatedly attaching and re-attaching the same mixing tip.
- Dispose of used tips and empty syringe properly.
- Verify flow prior to applying intraorally. If resistance is met, replace tip and re-check.
- UltraTemp® REZ should be refrigerated to maintain shelf life.
- Keep away from eyes.
- When using UltraTemp®/UltraTemp® REZ do not remove mixing tip until next use. Disinfect and wipe syringe and mixing tip with an intermediate level disinfectant between uses. Change mixing tip before each use.

NOTE: For MSDS and additional information about using UltraTemp® or UltraTemp® REZ, please go to www.ultradent.com

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