## Ref. 080407 (M18) / Ref. 080405 (M9)

ES .....	POLIMERIZADORA MAGIC .....	4
EN .....	POLYMERISER MAGIC .....	7
FR .....	POLYMÉRISEUSE MAGIC .....	10

**MESTRA®**

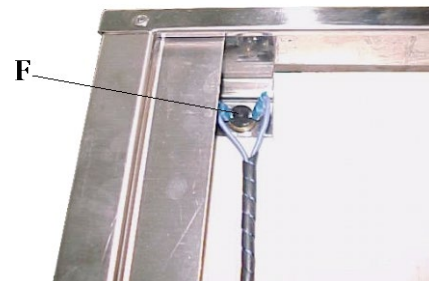
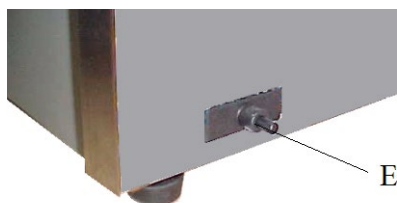
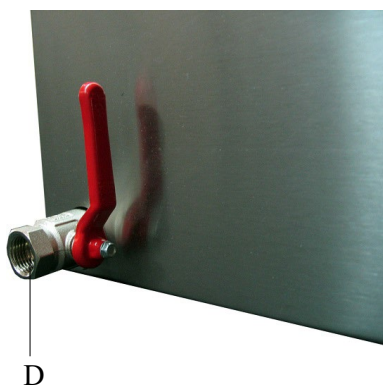
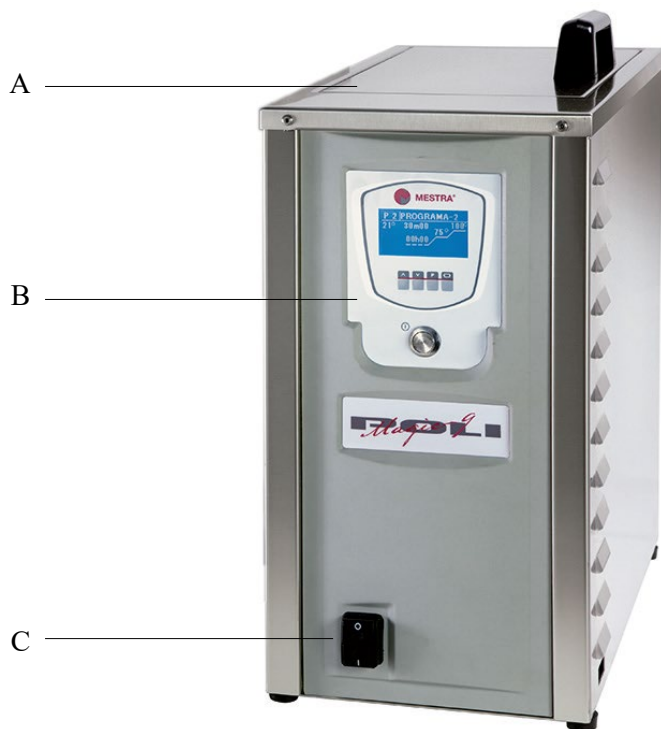
TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

A	Tapa / Lid
B	Panel de control / Control pannel
C	Interrupitor general / Main switch
D	Grifo de desagüe / Drain tap
E	Disyuntor / Circuit breaker
F	Contactador / Contactor

**^** Subir  
**Up**  
*Monter*

---

**v** Bajar  
**Down**  
*Descendre*

---

**P** Memoria  
**Memory**  
*Mémoire*

---

**↕** Cambio de display  
**Display change**  
*Changement de display*

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Ref. 080405 / Ref. 080407 is a practical polymeriser with a control panel on which sequences of polymerising times and temperatures can be programmed as required by the user.

## INSTALLATION

- Carefully unpack the polymeriser, removing the packing material inside the tank. Inside the tank you will also find a muffle tray.
- Set the unit up on a flat, horizontal, firm surface away from sources of heat or vibration. Ensure that there is a gap of at least 15 cm between the ventilation grilles and the nearest wall or other object.
- Connect the unit to a 230 V, 50/60 Hz earthed power socket protected by a suitable thermal magnetic cut-out. Consult your installer.

## USER CONFIGURATION

- From the home screen, press and hold the **P** key.

### CUSTOMER CONFIG

1.0 User

- Briefly press the key **P** again to go to the option *1.0 User*.

1.0 User

TWARN. = 10 ''  
 LANGUA = English  
 UNITS = °C

- Briefly press the key **P** to select:

**TWARN.:** Time for the warning of cycle end.  
**LANGUA:** Spanish, English, German, French.  
**UNITS:** Celsius, Fahrenheit

- Use keys **▲▼** to modify the values and the key **F** to save.

## PROGRAM EDITION

- Press the **P** key to access the program data.
- Use the key **s▲▼** to modify the values and the **P** key to save and go to another parameter:

A.T: Delayed start time (so that the machine starts working after some time has passed).

r1: Time to reach the 1st temperature

C1: First temperature

t1: Maintenance time of the 2nd temperature

r2: Time to reach the 2nd temperature

C2: First temperature

t2: Maintenance time of the 2nd temperature

In case of selecting the "FREE" value in a time, it implies that it is indefinite and that only the user can end said timing.

## PROGRAM SELECTION

- Being on the program number we can select another by pressing the keys **▲▼**.

## OPERATION

- Place the muffle tray in the bottom of the tank.
- Fill the tank with water up around 10 cm below the rim. There should always be at least 20 cm of water in the tank, to prevent damage to the heating element.
- Place the muffles on the muffle tray, using long, sturdy gauntlets to protect against splashes, a face-mask and muffle tongs. TALLERES MESTRAITUA, S.L. recommends MESTRA tongs. Close over the cover.
- Switch on main switch (C). Some of the controls on the panel will flash, indicating that the polymeriser is ready for operation.
- Program the unit as indicated above to ensure that the programs stored meet your own requirements. These programs can be used and modified as required in future.

- Select the program you want to run.
- Press the START/STOP button (C) to start the cycle.

- If need, press long the key ▲ to jump to the following step.
- Once the polymerisation program is completed, the unit disconnects automatically. If alarm is activated, it will sound at the end of the program (look at the Alarm section). To remove the muffles, open the cover carefully, as steam pouring out from the tank could cause burns. Use long gauntlets and a face-mask, and remove muffles using muffle tongs. Close over the cover.
- If there is a power cut, the unit retains in its memory the parameters of the step in progress, and when power is restored the program will continue from the point at which it stopped. If the switch is turned off before the program is completed, the furnace takes this as a power cut.
- Turn the main switch off to disconnect the unit from the power supply.
- Disconnect the unit via main switch (C). The fan remains on until switch is disconnected to prevent the control module from overheating.

## ERROR MESSAGES

TEMP. INTERNAL	An overheating of the electronics has been detected. Get in contact with your distributor.
TEMP. HIGH	The safety temperature has been exceeded.
PROBE FAIL	Indicates temperature probe failure. If the problem persists, contact your dealer.
POWER SUPPLY FAILURE	The machine warns that there has been a prolonged power outage. Although it will try to recover the program, it could have affected the work
TRIAC ERROR	Get in contact with your distributor.

## MAINTENANCE AND CLEANING

To obtain long, trouble-free service from the unit, follow these instructions:

- The unit has a circuit breaker that protects the machine against electric overloads. Once the problem is solved, press the circuit breaker located on the side of the machine (E) to get it into work again.
- The unit has a circuit breaker that protects the machine against accidental overheating as it detects excessive temperature and cuts off machine operation. Once the machine returns to normal temperature, the security contactor has to be reset. To access this, proceed as follows:
  1. Take off the lower panel by removing the fastening screws.
  2. Unscrew and extract the front of the machine holding the control unit.
  3. Press the contactor (F).
  4. Follow these instructions in reverse order to reassemble the unit.
  5. Carry out a test to check operation of the polymeriser.
- Clean the inside of the tank approximately every 20 hours, by heating the water to 75 °C, then emptying the tank through the drainage tap at the back.
- Unplug the polymeriser and clean the inside of the tank with a fine scourer moistened in soapy water, taking care not to damage the heating element or temperature probe. Special products for cleaning polymerisers may also be used. Rinse the inside of the tank with plenty of water and then close the drainage tap.
- At intervals which will depend on how hard the water is, clean away any limescale which may have been deposited on the shower nozzles or electric heating element, using a scale remover. Anti-limescale products may also be used.
- To clean the outside housing of the unit use a cloth moistened in soapy water. Do not use solvents or other inflammable products.
- To get the best results from the polymeriser additives available on the market to dissolve wax may be added to the water. The manufacturers' recommendations should always be followed in this case.
- After using the polymeriser, remove wax, foam, remains of resin and other particles which may be left floating on the water.
- For optimum performance, TALLERES MESTRAITUA, S.L. recommends the use of MESTRA muffles and flanges.
- Clean the unit's control panel regularly with a damp cloth. Always disconnect the machine before doing this.

## PRECAUTIONS

- Before connecting the unit ensure that the power supply is 230 V, 50/60 Hz and earthed. It should also be protected by a thermal magnetic cut-out. Consult your installer concerning these matters.
- Do not allow children or unskilled personnel to handle the unit.
- Use long gauntlets and muffle tongs whenever you put muffles into the tank or take them out.
- Open the cover carefully: steam pouring out from the tank could cause burns.
- Always keep the unit's cover closed while polymerisation programs are running.
- Never place more than 9 muffles on the polymerising tray.
- Always keep at least 20 cm of water in the bottom of the tank, as the heating elements may be damaged if the level is lower.
- Always leave a gap of at least 15 cm between the unit's ventilation grille and the nearest wall or other object.
- Before plugging in the unit, fill the tank with water to 10 cm below the cover.
- TALLERES MESTRAITUA, S.L. recommends polymerising at around 95 °C. At higher temperatures too much steam may be generated.

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**GUARANTEE**

All **MESTRA®** units are guaranteed against every defect in manufacturing for a period of one year from date of purchase. Prior to honouring a guarantee, **MESTRA®** will require that the attached coupon be filled in with date of sale, duly stamped by the distributor.

The guarantee does not cover breakdowns caused by incorrect installation or use.

**MESTRA®** will not effect any repair outside its own workshops and will require transport costs.

Repairs carried out on the premises of the customer, be they with or without guarantee, are entrusted to the technical assistance of the distributor who has sold the unit or alternatively to the technicians he has contracted

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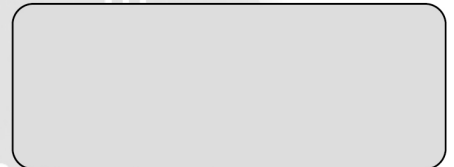
*TALLERES MESTRAITUA S.L. is a company manufacturing tools and appliances for the dental prosthesis sector, from the business address shown at the bottom of the page.*

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*I HEREBY STATE: That the MAGIC POLIMERISERS Ref. 0804054, Ref. 080407, with the following characteristics:*

	<b>Ref. 080405</b>	<b>Ref. 080407</b>
Alto x ancho x fondo / <i>Height x width x depth:</i>	420 x 270 x 480 mm	510 x 420 x 500 mm
Dimensiones cuba / <i>Tank dimensions:</i>	335 x 180 x 350 mm	350 x 320 x 400 mm
Peso / <i>Weight:</i>	15 kg	20 kg
Tensión / <i>Power supply:</i>	230 V, 50/60 Hz	230 V, 50/60 Hz
Consumo / <i>Consumption:</i>	1750 W	3000 W
Nº muflas con brida / <i>Nº muffles with flange:</i>	9	18

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*Meet the low voltage requirements set out in Directive 2014/35/UE and in Directive 2014/30/UE concerning electromagnetic compatibility.*

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*In witness whereof, the company issues this declaration of conformity in Sondika, on 05 July 2022.*

Firma / *Signed:*

Ignacio Mestraitua,  
Director Gerente / *Managing Director*

**MESTRA®**

**TALLERES MESTRAITUA S.L.**

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

**48150 SONDIKA - BILBAO - ESPAÑA**