· Conscious dentistry ·

VIDU

RETRACTION CORD
ES · HILO RETRACTOR
IT · FILO RETRATTORE
FR · FILO DE RÉTRACTION
PT · FIO DE RETRACÇÃO
DE · RETRAKTIONSFÄDEN

**"THE FUTURE
IS IN YOUR HANDS"**

VIDU
RETRACTION CORD

EN

INSTRUCTIONS FOR USE

Retraction Cord VIDU is a knitted non impregnated retraction cord made of 100% cotton.
Retraction Cord VIDU is indicated to accelerate the hardening of materials in dental treatments.
Retraction Cord VIDU comes in 6 thicknesses and a length of 254 cm.

INSTRUCTIONS FOR USE:

- How to prepare before use
1. Make sure it is the same device you intend to use.
 2. Be careful not to mix contaminations during use.
 3. Please note the manual from the manufacture and follow the instructions.
 4. Do not use in case of damaged protective.

OPERATING INSTRUCTIONS:

1. Press slightly into gums using the device.
2. Widen the space between gums and tooth using the floss.

PRECAUTIONS:

1. Usage should be only for dental specialist and doctor on the instructions.
2. Do not use when discolored or if there are impurities in the product.
3. Children and seniors should keep out of use.
4. Do not use for anything other than dental treatment purposes.

CAUTIONS:

Do not reuse. Let the expert operate the product.
Do not let children touch.
There is a risk of infection when reused.

INSTRUCTIONS FOR DISPOSAL:

The product should be discarded after use.

STORAGE:

Close the cap after use and store in dry place.

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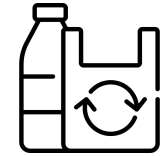
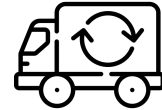
**SMART
WASTE**

**ONLINE
DISTRIBUTION**

Distributed by:
DENTARED ODONTOLOGY SERVICES
C. Torres Quevedo 13 Alcobendas,
28108, Madrid, Spain
www.vidu.eco

Made by:
ATRIA CO.
A-#701-712, Seoulsup SK V1 Tower, 5,
Seongsu-ro 8-gil, Seongdong-gu,
Seoul, Korea (South)

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