# VIDU

· Conscious dentistry ·

## ADDITION CURING SILICONE Putty Soft

**ES** - SILICONA DE CURADO POR ADICIÓN Putty Soft  
**IT** - SILICONA A POLIMERIZZAZIONE PER ADDIZIONE Putty Soft  
**FR** - SILICONA PAR ADDITION A POLYMERISATION Putty Soft  
**PT** - SILICONA POLIMERIZÁVEL POR ADIÇÃO Putty Soft  
**DE** - ADDITIONSVERNETZENDES SILIKON Putty Soft

**"THE FUTURE  
IS IN YOUR HANDS"**

VIDU


EN

## ADDITION CURING SILICONE Putty Soft


Polyvinylsiloxane for high precision dental impressions, addition-curing. Product complies with ISO 4823. For professional use. Rx ONLY.

### INSTRUCTIONS FOR USE

**APPLICATIONS:** Preliminary impressions for the two-step putty/wash and base for the sandwich technique. The impression can be electroplated.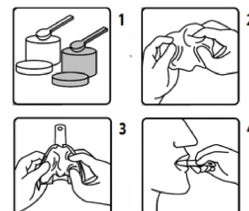

**PREPARATION:** Dry the areas to be reproduced and prevent further inflow of liquid into them. Measure out the two pastes in a ratio of 1:1, using the corresponding coloured scoops (1). Mix the two components vigorously to obtain a homogeneously coloured mass (2). Place the mixed mass on the impression tray (3) and insert it into the mouth (4).

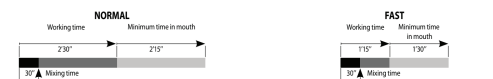TECHNICAL DATA	PUTTY SOFT NORMAL		PUTTY SOFT FAST		PUTTY HARD NORMAL		PUTTY HARD FAST	
	Type 0		Type 0		Type 0		Type 0	
ISO 4823 classification	Type 0		Type 0		Type 0		Type 0	
Mixing time	30"		30"		30"		30"	
Working time *	2' 30"	1' 15"	2' 30"	1' 15"	2' 30"	1' 15"	2' 30"	1' 15"
Minimum time in mouth	2' 15"	1' 30"	2' 15"	1' 30"	2' 15"	1' 30"	2' 15"	1' 30"
Recovery from deformation	99,9%		99,9%		99,9%		99,9%	
Strain in compression	4%		4%		1,9%		1,9%	
Linear dimensional change at 24 hours	-0,02%		-0,02%		-0,02%		-0,02%	
Detail reproduction	25µm		25µm		25µm		25µm	
Hardness Shore A	50		50		63		63	

\*Measurement at 23 +/- 2°C and 50 +/- 10% relative humidity.

**WARNINGS:** The above times can be influenced by:

- variations in the temperature of the material, the environment and/or the operator's hands during mixing (higher temperatures reduce them while lower temperatures prolong them)
- changes in the speed and/or force employed during mixing
- humidity
- Avoid direct or indirect contact with latex rubber dams, gloves or with components of such material. Contact with latex inhibits the hardening of polyvinylsiloxanes. Accurately wash your hands and use vinyl/nitrile gloves. Do not invert the lids of the containers and use the corresponding coloured scoop for each paste. Close the containers well after use. To eliminate traces of saliva or blood and guarantee optimal adhesion of the wash impression to the preliminary impression, rinse under running water and dry well. The impression can be stored without any special precautions. Cast the model in plaster at least 30 minutes after removal from the mouth.

For disinfection, immerse the impression for 5 minutes in a solution of glutaraldehyde (concentration 0.5%-2%) or sodium hypochlorite (concentration 0.1%-1%). Rinse the impression under running water after disinfection.

**IMPRESSION TRAY:** All types of rigid impression trays can be used.

**GUARANTEE:** The manufacturer guarantees the quality of its products and undertakes to replace them whenever defects in production are found within the expiry dates. The manufacturer will not be held responsible and will not recognize any damage that is direct, indirect, and accidental or caused by incorrect use of the material. Check the suitability of the product prior to use.

**STORAGE:** Shelf life 3 years when stored at 77 °F (25 °C).

**PRESENTATION:**

- [COD V152001] ADDITION CURING SILICONE Putty Soft, Normal Set
- [COD V152002] ADDITION CURING SILICONE Putty Soft, Fast Set.
- [COD V152003] ADDITION CURING SILICONE Putty Hard, Normal Set
- [COD V152004] ADDITION CURING SILICONE Putty Hard, Fast Set.

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