· Conscious dentistry ·

SALIVA EJECTORS

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INSTRUCTIONS FOR USE

USE DESTINATION:

These medical devices are used only in dental institutes from a specialized doctor or from qualified staff, during the various operations that they had to do; they are used to extract the saliva from the mouth of the patient. The saliva ejectors are composed of a PVC tube, with a copper-coated wire, and with a perforated stopper at one of the ends of the saliva ejector (the other end has to be connected to the aspiration machine). The peculiarity of this product is that it can be simply modelled from the user, because of the copper-coated wire: it can be modelled in the shape that the user finds most useful for the actual operation. They are supplied in multiple non-sterile packaging.

Fig.1:
Saliva Ejector before the modelling

Fig.2:
Saliva Ejector after the modelling

INSTRUCTIONS FOR USE:

- 1.- Take the saliva ejector, verify the safety and the functionality of the device and assure yourself that the ejectors are integral; if the ejector has a fixed tip verify that it is effectively fixed.
- 2.- Bend the saliva ejector to your specific needs, and pay attention to not crush too much of the tube, and check if something undesired happened (discharge of the inner wire, loss of the tip or something else).
- 3.- Put the saliva ejectors in the specific place on the tube of the aspirator.
- 4.- Put the saliva ejectors into the mouth of the patient and make sure to use it with care and diligence.
- 5.- If you are using a saliva ejector with a removable tip pay attention to the accidental detachment of it during the various operations inside the mouth of the patient.

The images given are only examples and they are not exhaustive.

STORAGE/SHELF-LIFE:

8 years from production period.

Way of conservation: far from direct heat.

PRESENTATION:

Packaging 100 pieces in plastic bag.

- [Ref. VI18000] Transparent	- [Ref. VI18005] Lila
- [Ref. VI18001] White	- [Ref. VI18006] Orange
- [Ref. VI18002] Blue	- [Ref. VI18007] Yellow
- [Ref. VI18003] Green	- [Ref. VI18008] Pink
- [Ref. VI18004] Lima	- [Ref. VI18009] Blue Dark

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- [Ref. VI18000] Transparente	- [Ref. VI18005] Lila
- [Ref. VI18001] Blanco	- [Ref. VI18006] Naranja
- [Ref. VI18002] Azul	- [Ref. VI18007] Amarillo
- [Ref. VI18003] Verde	- [Ref. VI18008] Rosa
- [Ref. VI18004] Lima	- [Ref. VI18009] Azul Oscuro

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