· Conscious dentistry ·

VIDU

DENTAL BONDING

en · Universal bond
es · Universal-Bond
it · Bonding universale
fr · Bond universel
pt · Bond Universal
de · Universal-Bond

"THE FUTURE IS IN YOUR HANDS"

VIDU
DENTAL BONDING

en

INSTRUCTIONS FOR USE

Patient target group

Children, adolescents, adults and older patients

Intended users

Dentist

Composition

Qualitative data: Oligomeric methacrylates, HEMA, camphorquinone
Quantitative data in % G/G: Methacrylates 98, Additive 2

Intended purpose of the product

DENTAL BONDING VIDU is an adhesive with excellent adhesion to dentin and etched enamel. It improves the adhesion between composite and tooth surface.

Indications

DENTAL BONDING VIDU is used for improved adhesion of composites to enamel and dentin. It also improves the adhesion between glass ionomer underfilling cements and composite fillings ("sandwich technique").

Contraindications

DENTAL BONDING VIDU must not be used in the pulp area.

Performance characteristics of the product

Curing by irradiation with light.

Specific precautions

None

Application

The cavity should be prepared according to the rules of the adhesive filling technique. Depending on the situation, calcium hydroxide paste or an underfilling cement should be placed in deep cavities. According to the etch-bond technique, the enamel is etched, rinsed and then dried. Glass ionomer cement filling should not be etched, but only roughened. After the prepared cavity is dry, 1-2 drops of light-curing DENTAL BONDING VIDU are dropped onto the mixing block (reseal the bottle immediately) and, using a brush or foam pellets, laid down in a thin layer. Excess is removed with a dry brush or a pellet. This is followed by curing with halogen light (15 seconds). Light devices should emit at 450 nm at 500 mW/cm² and should be checked regularly.

Warnings

Residual risks

The processing instructions and precautionary measures described in the instructions for use of the product must be observed in any case. If, despite following these instructions, an adverse reaction occurs in the patient or other adverse events occur, we request that you inform us in every case, describing the accompanying circumstances as precisely as possible.

Side effects

Undesirable side effects of this medical device, when the device is properly processed and used, are extremely rare. If you become aware of any adverse effects – even in doubtful cases – please inform us and describe the accompanying circumstances and symptoms as precisely as possible.

No risks in the event of an overdose are known of.

Interactions

Interactions of the product with other medical devices, medicinal products or other substances is not known.

Storage

Store DENTAL BONDING VIDU in a cool, dry place and protect from light. The product should be stored between 0°C -22°C.

Minimum shelf life

3 years

Do not use after the expiry date

Variants

[REF] V191001 1 x 5,0 ml liquid

Disposal

Unused material can be cured and disposed of with household waste. Uncured material should be disposed of as methacrylate waste.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

		H315 causes skin irritation
		H317 may cause allergic skin reactions
		H319 causes serious eye irritation
		H335 may cause respiratory irritation
		H411 toxic to aquatic organisms, with long-lasting effect

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