· Conscious dentistry ·

VIDU

FLOWABLE COMPOSITE

- en** · Light curing small hybride-composite (all purpose)
- es** · Composite fluido micro-hibrido fotopolimerizable
- it** · Fotopolimerizzabile per piccole superfici di composito ibrido
- fr** · Composite hybride photopolymérisable, fluide (por tous usage)
- pt** · Composito micro-hibrido flúvel fotopolimerizável
- de** · Lichthärtendes Feinglas-Hybridcomposit (universell einsetzbar)

“THE FUTURE IS IN YOUR HANDS”

VIDU

FLOWABLE COMPOSITE

en

Instructions for use

Patient target group
Children, adolescents, adults and older patients

Intended users
Dentist

Composition
Qualitative composition: 1,4-butanediol-dimethacrylate, Bis-GMA, aliphatic urethane dimethacrylate, butylated hydroxytoluene, camphorquinone, silicon dioxide, glass ceramic, iron oxide pigments, 2-(2H-Benzotriazol-2-yl)-p-cresol, 2-ethylhexyl-4-(di-methyl-amino) benzoate, diphenyl (2,4,6-trimethylbenzoyl) phosphine oxide
Quantitative composition: 1,4-butanedioldimethacrylate 5 %, aliphatic urethane dimethacrylate 22 %, Bis-GMA 18 %, silicon dioxide 6 %, activators and UV absorber <1 %, glass ceramic 48 %, pigments <1 %

Intended purpose of the product
FLOWABLE COMPOSITE VIDU is a light-curing, radiopaque, free-flowing composite filling material.

Indications
Cavities of classes III, IV and V.

Contraindications
Do not use in patients with known allergy to the ingredients.

Performance characteristics of the product
FLOWABLE COMPOSITE VIDU is a free-flowing composite that cures with light.

Specific precautions
None

Application
1. Clean and dry the preparation. Determine the colour.
2. Prepare the cavity according to the general rules of adhesive technique. Bevel the edges of the enamel.
3. Apply etching gel to the enamel edges for 20 to 35 seconds.
4. Completely rinse off the etching gel.
5. Dry with oil-free air. Avoid over-drying the dentin.
6. Apply adhesive as described in the usage instructions.
7. Select the syringe in the desired colour. Remove the syringe cap. Screw on the Luer Lock canula.
8. FLOWABLE COMPOSITE VIDU is applied directly by pressing the syringe.
Apply FLOWABLE COMPOSITE VIDU in layers no thicker than 2 mm to ensure complete curing.
9. Leave each layer to light cure for 30 to 40 seconds.
10. Repeat steps 8 and 9 until the cavity is filled.
11. Remove excess, contour and finish with diamond finishers and flexible discs of decreasing grain size.

Warnings
Residual risks
Please observe carefully
Light units should be emitting at 450 nm at 500 mW/cm2 and should be checked regularly. Place the light as close as possible to the filling material. Do not remove the oxygen-inhibited layer, as this is needed for bonding with the next layer. Using a die is recommended. Follow the instructions for use of other materials used with FLOWABLE COMPOSITE VIDU.

Safety information
Use is contraindicated if drainage or the prescribed application method is not possible. Do not use directly on opened pulp or if there is a known allergy to any ingredient. No side effects have been reported to date.
In individual cases, hypersensitivity to components of the material may occur. Avoid skin contact with the paste. In case of accidental contact, wash immediately with soap and water. In case of eye contact, rinse immediately with plenty of water and consult a doctor. Avoid materials containing eugenol, moisture and oily air, as they can prevent polymerisation at the point of contact. Discolouration may occur when using cationic mouthwashes, plaque disclosers or chlorhexidine.

Side effects
Adverse side effects are not known.
Risks in case of overdose are not known.

Interactions
Interaction of the product with other medical devices, medicinal products or other substances is not known.

Storage
The product should be stored between 0°.

Minimum shelf life
Do not use after the expiry date

Variants
[REF](#) V192101 1 x syringe A1 with 1,0 ml + cannulas
[REF](#) V192102 1 x syringe A2 with 1,0 ml + cannulas
[REF](#) V192103 1 x syringe A3 with 1,0 ml + cannulas
[REF](#) V192104 1 x syringe A3,5 with 1,0 ml + cannulas
[REF](#) V1921051 x syringe B1 with 1,0 ml + cannulas
[REF](#) V192106 1 x syringe B2 with 1,0 ml + cannulas
[REF](#) V192108 1 x syringe C2 with 1,0 ml + cannulas

Disposal
Unused material can be disposed of with household waste after curing.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

 H317 May cause allergic skin reactions
H318 Causes serious eye damage

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