· Conscious dentistry ·

# VIDU

## TEMPORARY FILLING

**es** · CEMENTO PROVISIONAL  
**it** · OTTURAZIONE PROVVISORIA  
**fr** · OBTURATION PROVISOIRE  
**pt** · OBTURAÇÃO PROVISORIO  
**de** · PROVISORISCHE VERSCHLUSS

“THE FUTURE  
IS IN YOUR HANDS”

# VIDU

TEMPORARY FILLING MATERIAL VIDU

en

### INSTRUCTIONS FOR USE

TEMPORARY FILLING MATERIAL VIDU is a proven, ready-to use close-up-material.

It guarantees reliable sealing due to the following properties:

- rapid curing in the cavity;
- strong adhesion to dentin;
- outstanding marginal seal;
- resistant to chewing pressure

TEMPORARY FILLING MATERIAL VIDU damages neither the pulp nor the gums.

#### TYPE OF APPLICATION:

Use a filling instrument to apply the required quantity of TEMPORARY FILLING MATERIAL VIDU in the moist cavity. Curing begins after a few minutes. We recommend protecting the exposed location for 15 minutes. Deep cavities do not have to be completely filled, which makes subsequent removal easier. When sealing inlays, as far as is possible, ensure pressure-free application.

A pointed, short and strong probe is best suited for removing the set TEMPORARY FILLING MATERIAL VIDU. Insert it between the dentin and filling.

#### SIDE EFFECTS:

Undesirable side-effects are extremely rare if the product is prepared and used properly. Local dysesthesia (e.g. taste irritations or irritations of the oral mucosa) can however, not be completely precluded. Should you become aware of any undesirable side effects, including in cases of doubt, please notify us, giving as precise a description of the attendant circumstances and symptoms as possible.

#### WORKING INSTRUCTIONS/PRECAUTIONARY MEASURES:

Ensure that the working instructions and precautionary measures described in the supplied product information are always strictly observed. Should the patient suffer undesirable reactions or any other unwanted incidents arise even though these instructions were observed, then please inform us and give a precise description of the accompanying circumstances.

#### STORAGE/SHELF-LIFE:

Always close container immediately after removing material. Shelf-life of the closed packaging: See label. The preparation can be stored for at least 6 months after being opened for the first time. Do not store above 22°C. Do not use after expiry of the use-by date.

#### PRESENTATION:

- REF: V193001 Jar x 30 gr
- REF: V193002 Jar 2 x 30 gr

Keep away from children!  
For professional dental use only!

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