# VIDU

· Conscious dentistry ·

## GLASS FIBER POST

POSTE DE FIBRA DE VIDRIO  
PERNI IN FIBRA DI VETRO  
TENON DES FIBRE DE VERRE  
PINO EM FIBRA DE VIDRIO  
GLASFASERVER WURZELSTIFT

**"THE FUTURE  
IS IN YOUR HANDS"**

**VIDU**  
GLASS FIBER POST

EN

### INSTRUCTIONS FOR USE

Endocanal posts with a translucent, radiopaque glass fibre conical tip. The radiopacity of the post allows it to be easily distinguished from the cement. GLASS FIBER POST VIDU posts are made of glass fibre and epoxy resin.

#### SIZES:

Glass Fiber Post VIDU (19 mm length) and Post Drill VIDU are available in 4 color-coded sizes for corresponding use:

Glass Fiber Post VIDU		Post Drill VIDU
Colour code	Medida Ø	Colour code
White	1,0	White
Yellow	1,2	Yellow
Red	1,4	Red
Blue	1,6	Blue

#### INDICATIONS:

Glass Fiber Post VIDU posts are indicated for all clinical cases. Sequence of clinical use

1. Use the post at the end of the root canal treatment.
2. Remove the root canal filling up to 3-4 mm from the apex.
3. Choose the post which should be as large as possible and compatible with the diameter of the canal:  
- The part that enters the canal should be at least as long as the part that will remain outside after restoration.
4. Once the post is chosen, enlarge the channel with the appropriate Post Drill VIDU, see table. Post Drill VIDU burs must be sterilized before use.
5. Irrigate the canal with endodontic solutions (NaOCl 5%.);
6. Procedures: Check the adaptation of the post in the canal: it should reach the depth established before.

- If necessary, cut the coronal part of the post to the required length. Use only diamond rotary instruments. Perform this operation away from the patient wearing gloves, mask and goggles.  
- Clean the post by immersing it in ethyl alcohol for 60 seconds or in 10vol hydrogen peroxide; then wash with water spray and dry.  
- Glass Fiber Post VIDU posts can be used with adhesive systems and with dual self-setting cements, follow the manufacturer's recommendations.

#### PRECAUTIONS:

For Material Safety Data Sheets (MSDS), go to [www.vidu.eco](http://www.vidu.eco)

#### STORAGE:

The product should be stored in a cool, dry place protected from sunlight.

#### DISPOSAL:

Dispose of in accordance with current regulations.

#### WARNINGS AND CONTRAINDICATIONS:

Do not use in patients with known or suspected allergies to materials constituting the device.

Glass Fiber Post VIDU can be sterilised in pouches using autoclave at 135°C and 2.1 bar pressure for 5 minutes.

- Keep away from children
- Product intended for dental use only.

#### PRESENTATION:

- [Ref. V195002] GLASS FIBER POST No 1.0
  - [Ref. V195003] GLASS FIBER POST No 1.2
  - [Ref. V195004] GLASS FIBER POST No 1.4
  - [Ref. V195005] GLASS FIBER POST No 1.6
  - [Ref. V195000] GLASS FIBER POST Kit
- Kit Contents 5 posts + 1 drill bit in each size 1.0, 1.2, 1.4 and 1.6

Note: This material is developed for use in the dental field only. Its use must be in accordance with the related instructions. The manufacturer assumes no liability for damage resulting from non-compliance with the instructions for use or for uses other than the intended field of application of this product. Users are responsible for testing the material in fields of use not explicitly indicated in the instructions for use. Descriptions and data do not constitute any guarantee for the properties of the products and are not binding.

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