· Conscious dentistry ·

VIDU

PIT AND FISSURE SEALER

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en

INSTRUCTIONS FOR USE

PIT AND FISSURE SEALER VIDU is a light-curing composite with fluoride for fissure sealing. It is used to seal fissures and occlusal surfaces for the purpose of caries prophylaxis.

Application

1. Thoroughly clean the surface to be sealed and coat with etching gel (orthophos-phoric acid). The blue colour makes it easy to apply the gel accurately and uniformly. Allow the etching gel to act for approx. 60 seconds. Shorten the etching time for the enamel of teeth of young people to approx. 30 seconds, and extent to approx. 120 seconds for prism-deficient or fluoridised enamel. Rinse thoroughly with plenty of water and then dry thoroughly the surface that has to be sealed well. The etched tooth surfaces should have a milky-white appearance. After drying avoid contact with liquids. 2. Apply a single drop of PIT AND FISSURE SEALER VIDU on a mixing pad and brush a thin layer on the etched surfaces. Cure for 20-30 seconds with a halogen lamp. After use, tightly close the syringe.

Side-effects

Provided that this medical product is correctly processed and applied, undesirable side-effects are very rare. In the event that undesirable side-effects do appear - even when in doubt - please inform us and give a precise description of the accompanying circumstances and symptoms. We will definitely follow up all notifications.

Contra-Indication

Allergies against acrylates.

Application remarks

The application instructions and other precautionary measures described in the directions for the use of this product must be observed. If patients still develop undesirable reactions or other effects, even though these instructions have been carefully observed, please notify us in all cases and give a precise description of the accompanying circumstances.

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