· Conscious dentistry ·

MTA UNIVERSAL

en · Biocompatible endodontic repair cement
es · Cemento de reparación endodóntica biocompatible
it · Cemento di riparazione endodontica biocompatibile
fr · Ciment de réparation endodontique biocompatible
pt · Cimento de reparação endodôntica biocompatível
de · Biokompatibler endodontischer Reparatur-Zement

	en
INSTRUCTIONS FOR USE	

PRODUCT DESCRIPTION: MTA-Universal is a biocompatible endodontic repair cement in DirectCaps. MTA-Universal powder consists of very fine hydrophilic particles of several mineral oxides. After contact with MTA-Universal liquid it forms a gel that hardens to an impermeable barrier. MTA-Universal is delivered in DirectCaps. DirectCaps are easily activated and the content of the capsule is easily ejected out with the DirectGun. Capsule mixing (mixing time 30 seconds) is achieved by a high frequency mixer with about 4,300 oscillations/min such as Capmix.

INDICACIONES DE USO: Repair of root perforations (during root canal therapy), Root end filling (retrograde), Pulp capping, Apexification / Root end filling (orthograde), Repair of internal restoration, Pulpotomy
PERFORMANCE FEATURES: The performance features of the product meet the requirements of the intended use.

CONTRAINDICATIONS: None known
PATIENT TARGET GROUP: Persons who are treated during a dental procedure.
INTENDED USERS: This medical device should only be used by a professionally trained dental practitioner.

SIDE EFFECTS: None known

APPLICATION:
1. Activation and Mixing (see Instruction for DirectCaps)
Activate and mix the DirectCap according to the information in the DirectCap instruction. Mixing time for the DirectCaps is 30 seconds.
Attention: Avoid lag times between the processes of activation, mixing and application as the material is in the process of setting and lag times may impair or prevent application of the material. The material must be extruded within 10 seconds after the end of mix.
To prevent dehydration during setting, apply MTA-Universal intraoral immediately after mixing. Working time (at 23°C) of MTA-Universal is approx. 2:00 minutes from start of mix.

2.1. Repair of root perforations
Place rubber dam and clean the root canal system using intracanal instruments and NaOCl solution. Dry the root canal with paper points and isolate the perforation.
Fill the root canal apical of the perforation completely with a suitable root canal filling material. Mix MTA-Universal as described under point 1 and extrude it on a glass plate.
Apply MTA-Universal with suitable instruments into the perforation site and condense it. Check the position of MTA-Universal in the root canal by an X-ray. If an adequate barrier has not been created, rinse MTA-Universal out of the canal and repeat the procedure.
Remove excess moisture with a cotton pellet or a paper point. Place a cotton pellet in the access to the root canal and apply a temporary filling material. Alternatively seal the root canal with a suitable root canal filling material and seal the cavity with a tight filling.
Both options can be done at the earliest 5 minutes after placement of the MTA-Universal.
MTA-Universal repair material remains as a permanent part of the root canal filling.
2.2. Root End Filling (retrograde). Create a surgical access to the root-end and resect the root. Prepare an apical cavity to a depth of 3-5 mm.
Isolate the area and dry the root end cavity with paper points. Achieve hemostasis with suitable methods. Mix MTA-Universal as described under point 1 and extrude it on a glass plate. Apply MTA-Universal with suitable instruments and condense it using a small plunger. Remove excess cement and clean the surface of the root with a moist piece of gauze. Confirm placement of the MTA-Universal repair material with an X-ray.
MTA-Universal remains as a permanent part of the root canal filling.

2.3. Pulp Capping. Place rubber dam and prepare the cavity. Rinse the cavity and exposed pulpal areas with a suitable disinfectant. Mix MTA-Universal as described under point 1 and extrude it on a glass plate. With a suitable instrument apply a small amount of MTA-Universal over the exposed pulp and remove excess moisture with a cotton pellet. At the earliest 5 minutes after application of MTA-Universal place a small amount of a flowable light cure liner and light cure. Etch the remaining cavity walls according to the total-etch-technique with ETCHING GEL and apply a suitable bonding agent according to the corresponding instructions. Place a light cure composite according to the instructions and light cure. Check pulp vitality and status regularly.

2.4. Apexification / Root End Filling (orthograde)
Place rubber dam and clean the root canal system using intracanal instruments and paper points with NaOCl. Dry the root canal with paper points. Do disinfection place calcium hydroxide paste in the root canal and seal the access opening with a temporary filling material. After one week remove the calcium hydroxide paste from the root canal.
Mix MTA-Universal as described under point 1 and extrude it on a glass plate.
With a suitable instrument apply a small amount of MTA-Universal into the apical region and condense it. Create a 3 – 5 mm barrier of MTA-Universal. Check the position of MTA-Universal by an X-ray. If an adequate barrier has not been created, rinse MTA-Universal out of the canal and repeat the procedure. At the earliest 5 minutes after application of MTA-Universal place a cotton pellet in the access to the root canal and apply a temporary filling material. Alternatively seal the root canal with a suitable root canal filling material and seal the cavity with a tight filling.

ADDITIONAL NOTES/WARNINGS: In the first hour after application handle the placed MTA cement carefully. Store MTA-Universal in the sealed packaging in a dry place prior to use. MTA-Universal can cause discoloration. Keep away from children!
COMPOSITION: MTA, bismuth oxide
STORAGE: Store at 10-25 °C (50-77 °F) in the original seal-pack and protected from moisture. Do not use after expiry date. DirectCaps are for single use only.
DISPOSAL: Disposal of the product according to local authority regulations.
REPORTING OBLIGATION: Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.
Note: The summary of safety and clinical performance of the medical device can be found

VIDU

in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eu-damed>).
WARRANTY: The manufacturer warrants this product will be free from defects in material and manufacture. The manufacturer makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and the manufacturer's sole obligation shall be repair or replacement of the product.
LIMITATION OF LIABILITY: Except where prohibited by law, the manufacturer will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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**First Scientific Dental Materials GmbH**