OraSeal® Caulk & Putty

Rubber Dam Caulking & Block-Out Putty

Description:

OraSeal® Caulking and OraSeal® Putty are specially designed, cellulose-based caulking and block-out materials that are syringe delivered through a White Mac® tip. OraSeal Caulking will adhere to a wet rubber dam, gingival and mucosal tissues, and teeth. It will adhere under water and saliva!

OraSeal Putty is a heavier-bodied material with the same adhesive qualities as OraSeal rubber dam caulking.

Indications:

OraSeal Caulking is used to seal rubber dams to optimize tissue isolation; i.e., irritating seepage of strong peroxide solutions during power bleaching or hydrofluoric acid during porcelain etching, etc. It is excellent for repair of rubber dam leakage resulting from difficult adaptation associated with endodontic procedures, small perforations, and areas where fixed partial denture connectors prevent adequate seal. OraSeal Putty is used to block out undercuts associated with large gingival embrasures, implant bars, precision attachments, or fixed partial denture pontice during impression making procedures. It's heavier body prevents displacement during intraoral pick-up of anchoring attachments with autopolymerizing resins.

Procedure:


1.Remove luer lock cap from syringe and twist disposable tip securely onto syringe. Hold plunger in palm of hand for optimum control instead of conventional manner with thumb on plunger. Note: One may wish to custom trim tips to desired orifice diameter. OraSeal Caulking Uses:

- Rubber dam placement prior to vital bleaching, porcelain etching, etc.
- Improve dam seal adjacent to fixed partial denture connector.
- When performing endodontics on a compromised tooth.
- Torn rubber dam adjacent to hard and/or soft tissues or fixed prosthesis.
- To prevent saliva contamination when bonding ortho brackets.

OraSeal Putty Uses: Use particularly as a block-out material. Use before impression making under fixed bridgework, etc., or to block out under attachments and implant bars.

Precautions:


1. If surface treatment is to be performed following use of OraSeal, always make sure area is well cleaned with prophyl cup.
2. We recommend that large volumes removed not be vacuumed into vacuum system.
3. OraSeal is not to be expressed into extraction sites, deep pockets, etc., as one cannot be ensured of material being abraded away in such areas.
4. Clean and disinfect syringes between patients. We highly recommend using Ultradent® Syringe Covers sealed with the Ultradent® Impulse Sealer as a protective barrier to prevent cross-contamination.
5. OraSeal is NOT to be used in critical dimension areas. At any rate, always properly disinfect and wash impressions before pouring stone.

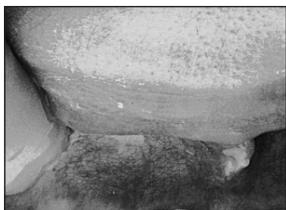1. To optimize rubber dam seal when procedure demands optimal isolation, run small bead of caulk just below free gingival margin, followed by repositioning the rubber dam over the caulking material.

2. Rubber dam repositioned with margins inverted and attached to frame in conventional manner. Clean all exposed tooth surfaces to ensure removal of OraSeal and any extraneous materials or stains with prophylaxis paste and cup.

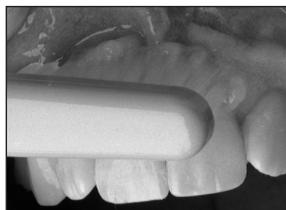3. Upon completion of the procedure, a wet toothbrush, floss, curette, and firm air/water rinse will quickly remove any remaining material.

- 1-3 Express OraSeal Caulking under fixed partial denture connector using a delivery tip. Shape with wet gloved finger, wet cotton swab or instrument. Root canal is performed in a clean, dry field.

1. A bead of OraSeal Caulking is run circumferentially around compromised tooth with heavier amount in area of most difficult adaption (in this case deep root decay on distal root) and rubber dam applied.

2. Root canal is performed in clean dry field with what otherwise (outside of more elaborate preparations) very well could have been a mess. Upon completion, make sure all OraSeal is removed from deep subgingival areas by thorough curetting and rinsing.

- Seals leak in rubber dam even when dam is submerged! Apply around border, then criss-cross over hole while expressing OraSeal Caulking until seal is completed.

- To ensure moisture control when bonding lower orthodontic brackets, seal embrasures with OraSeal Caulking or Putty to prevent saliva from sublingual salivary gland ducts from contaminating the area. Photo courtesy of Dr. Phillip Cambell, Huntsville, Texas.


1. Express against dry to damp tissue and under attachment with tip.

2. Shape with wet gloved finger or wet instrument.

3. Clean excess from top of attachment with wet 2x2 gauze. Area is now ready with undercuts blocked out. To remove, rinse and brush. Maintain vacuum.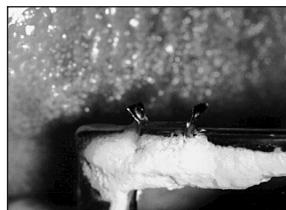

4. Bar attachment ready for anchoring with cold cure acrylic.

1. One may express directly from syringe to area under bar. Can also be reshaped and placed.

2. Molded, shaped, and contoured with wet thumb and fingers.

3. As with attachment, clean excess from top and sides of bar with wet 2x2 gauze. Bar ready for impression.

4. When finished, scrape away excess and discard in garbage. DO NOT RUN LARGE VOLUME THROUGH VACUUM LINES.

5. Again, as with attachments, clean remaining material with constant use of water, brush and vacuum.

1. For Professional use only.
 2. Review instructions, precautions, and SDS before beginning treatment. Use only as directed.
 3. Keep products out of heat or sunlight.
 4. Avoid skin exposure to resins.
- EN**
5. Isolate strong chemicals to area of treatment.
 6. Confirm that patient has no known allergies to treatment materials.
 7. Test flow of materials from syringe and tip before using intraorally.
 8. Never force syringe plungers.

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Health

Reactivity

HAZARD RATING
4 = Severe
3 = Serious
2 = Moderate
1 = Slight
0 = Minimal

0

0

0

0

U.S. federal law restricts the sale of this device by or on the order of a dentist.

Keep out of reach of children.

For immediate reorder and/or complete descriptions of UltraDent's product line, refer to UltraDent's catalog or call Toll Free **1-800-552-5512**. Outside U.S. call (801) 572-4200 or visit www.ultradent.com.

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Oraseal® Caulk & Putty
Rubber Dam Caulking & Block-Out Putty

Manufactured by:
UltraDent Products Inc.
500 West 100th Street
Mishawaka, IN 46545 USA
Made in USA

EGREP

UltraDent Products GmbH
Am Westhof Berg 30
51149 Cologne
Germany

For product SDS please see our website at www.ultradent.com
1-800-552-5512; 801-572-4200

ULTRADENT PRODUCTS, INC.

© Copyright 2011, UltraDent Products, Inc. All Rights Reserved. 10048-07 09/15