VIDU

INSTRUCTIONS FOR USE

The BOWIE-DICK VIDU TYPE TEST PACK consists of a lead-free steam-sensitive chemical indicator test sheet positioned in a package of layered porous materials. The porous materials are paperboard sheets that have moisture-imperious plastic layers at the top and bottom. These materials are wrapped in a non-woven, disposable wrap secured with a lead-free steam indicator label. The test sheet consists of a lead-free chemical indicator printed on paper as a yellow-colored diagonal pattern and is positioned near the center of the porous pack. The primary test sheet will turn a uniform grey/black color, except when air removal failures such as air leaks occur. An air removal failure is indicated as a lighter-colored area near the center of an otherwise dark-colored test sheet.

CONTRAINDICATIONS:

Do not use the BOWIE-DICK VIDU TYPE TEST PACK to monitor air removal from gravity displacement steam sterilizers, dry heat, ethylene oxide or other low temperature sterilization processes.

INDICATIONS FOR USE:

The BOWIE-DICK VIDU TYPE TEST PACK is designed to test the air removal efficiency of steam sterilizers with dynamic air removal of 132- 134°C (270- 273°F).

1. Insert a test sheet in the middle of the load cards and place them inside a stainless steel cassette.
2. In an empty autoclave, place the Bowie-Dick VIDU type test pack in the bottom tray, over the drain, at the front.
3. Run a sterilization cycle.
4. When the cycle is complete, remove the test pack (CAUTION: the test pack may be hot). Examine the BOWIE-DICK VIDU TYPE TEST PACK sheet - the test is positive if the chemical indicator changes to the correct color.
5. Record the necessary information on the sheet or in a log book and retain the records.

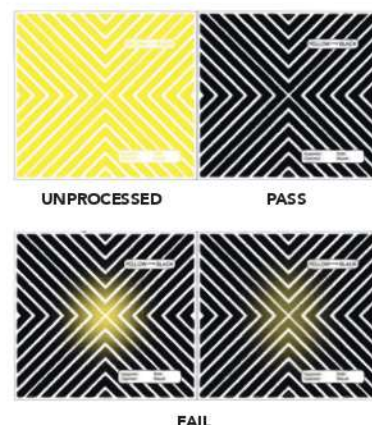
Note: Test the autoclave once a day, after installation or relocation, after a failure in the sterilization process or after repair. Charge cards can be used only 20 times. Please replace the load cards after 20 uses.

SHELF LIFE:

3 years from the date marked on the packaging

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