· Conscious dentistry ·

VIDU

CLORHEXIDINE SOLUTION

es · SOLUCIÓN DE CLORHEXIDINA
it · SOLUZIONE DI CLOREXIDINA
fr · SOLUTION DE CLORHEXIDINE
pt · SOLUÇÃO DE CLORHEXIDINA
de · CLORHEXIDIN -LÖSUNG

“THE FUTURE
IS IN YOUR HANDS”

VIDU

CLORHEXIDINE SOLUTION

en

INSTRUCTIONS FOR USE

Description

Root canal irrigant solution of 2% chlorhexidine digluconate.

Composition

Chlorhexidine Gluconate, Distilled Water

Recommended Indications

- It is used in root canal treatments, cleaning and moisturizing the canal by washing.
- It physically and chemically removes soft and hard tissue residues.
- Used in root canal irrigation and disinfection.

Contraindications

- In rare cases, individuals who are allergic to the components of the device may experience an allergic reaction. When such situations occur, stop using the product and consult a specialist.

Application

After removing the features smear layer with the combination of EDTA + NaOCl, the root canal is irrigated with 3 ml of distilled water. Irrigation with 5 ml of chlorhexidine solution per channel and wait 3-5 minutes. Drying is done.

Side Effects

There are no known side effects to date.

Storage

Do not use expired products. It should be stored in a cool environment, the product should come to room temperature before it is applied. Storage temperature should be between 1-30 ° C / 33.8 -86 ° F.

Disposal: Disposal must be made according to official regulations. Take to an approved landfill or a waste incineration plant, under conditions approved by the local authority.

Uncleaned packagings: Disposal must be made according to official regulations

Cautions

- Keep out of reach of children!
- For dental use only!
- It is irritating; Avoid contact with eyes, skin and mucous membranes.
- Never drink.
- In practice, the instructions for use should be followed.
- No liability will be accepted in cases where the specified usage area and usage instructions are not followed.
- The user is responsible for the results of the use and trials other than those specified in the instruction.
- Definitions and information do not constitute a guarantee nor are they binding.

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Ticaret Limited Sirketi