· Conscious dentistry ·

VIDU

ABRASIVE & POLISHING KIT

- en** · Composite polishing kit
- es** · Composite polishing kit
- it** · Composite polishing kit
- fr** · Composite polishing kit
- pt** · Composite polishing kit

en

INSTRUCTIONS FOR USE
DESCRIPTION: Dental rotary instruments: burs, abrasives and polishers; are made from one piece of hardened, coated stainless steel for dental clinic applications. The dental rotary instruments are supplied non-sterile.

INDICATIONS FOR USE: VIDU dental rotary instruments can be used for a wide variety of restorative procedures, including but not limited to: removal of caries or old restorations, preparation of cavities for restoration, finishing of restorations, preparation of crowns, orthodontic appliance procedure and any other dental operation. The pattern of the dental rotary instruments should be selected for the specific application. To obtain the best performance from the product, always adapt the rotary instrument to the task to be performed.

CONTRAINDICATIONS: The use of VIDU rotary instruments is contraindicated in any patient who is allergic to any of the product components.

Used VIDU rotary instruments should be considered contaminated and, as such, appropriate precautions should be taken during reprocessing and disposal. Appropriate eye protection, gloves and mask should be worn during reprocessing.

WARNINGS: Dental rotary instruments: burs, abrasives and polishers; are not sterile upon receipt and must be sterilized before use in accordance with the following instructions.

Dental rotary instruments can be damaged by alkaline and acidic detergents. Always use a pH neutral detergent or solution. Do not exceed 140°C. Remove all packaging prior to cleaning and sterilization.

Used dental rotary instruments should be handled with gloves and eye protection should be used.

LIMITATIONS ON REPROCESSING: Repeated processing has minimal effect on these dental rotary instruments. End of life is normally determined by wear and damage due to use. See Inspection.

INSTRUCTIONS:

Storage and Transportation: Store in a dry and clean place before use.

Protect delicate or fine dental rotary instruments from damage.

Point of Use: Remove from packaging. Remove all excess soil with disposable cloth paper wipe.

Preparation for cleaning: No specific requirements. Disassembly not required.

Automatic Cleaning:

Equipment: Use only validated washing machines and validated cleaning agents suitable for both devices and washing machine. Follow instructions issued by both the equipment and cleaning agent manufacturers. When unloading, inspect for complete removal of visible soil. If necessary, repeat cycle, or use manual cleaning before repeating cycle.

MAINTENANCE: Discard damaged dental rotary instruments.

INSPECTION AND FUNCTION TESTING:

All dental rotary instruments: Visually inspect to ensure that all contamination has been removed. Check for damage or wear. Cutting edges should be free of defects. Discard damaged or corroded instruments.

When instruments are part of a larger assembly, check the assembly with the corresponding components. There may be evidence of discoloration or dulling after sterilization. This is normal and will not affect performance.

DRYING: Dry using paper to weling or dry heat not exceeding 140°C.

PACKAGING FOR STERILIZATION: The UK Dept. of Health recommends that validated, pre-vacuum autoclaves are used for sterilizing wrapped dental rotary instruments. Autoclaves drawing

VIDU

only a post-sterilization vacuum for drying purposes are not suitable for processing wrapped dental rotary instruments. Dental rotary instruments may be loaded into dedicated instrument trays or general-purpose sterilization trays.

Ensure that cutting edges are protected.

STERILIZATION: Use only validated steam autoclaves.

Sterilization Temperature range (°C) at 134°C ,Nominal Pressure (MPa) 2.2,holding time 15-20 mins when nominal operating temperature and pressure have been reached.

ADDITIONAL INFORMATION: When sterilising multiple dental rotary instruments: diamond burs, carbide burs and other abrasives and polishers;in one autoclave cycle, ensure that the manufacturers stated maximum load is not exceeded. Validated ultrasonic baths may be used with validated cleaning agents. Follow instructions for use issued by both the equipment and cleaning agent manufacturers.

These instructions have been validated by the manufacturer as being capable of preparing a device for re-use.

It remains the responsibility of the reprocessor to ensure that the reprocessing has actually achieved the desired result. This normally requires validation and routine monitoring of the process. Any deviation by the reprocessor from these instructions should be properly evaluated for effectiveness and potential adverse effects.

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Shenzhen Dian Fong Abrasives Company Limited