R<sub>x</sub>

EN- Refrigerate  
DE- Kühlen  
FR- Réfrigérer  
NL- In de koelkast bewaren  
IT- Conservare in frigorifero  
ES- Refrigerar  
PL- Przechowywać w chłodziarce  
PT- Refrigerar  
SV- Kyl ned  
DA- Opbevares i køleskab  
FI- Jäähdytettävä.  
EL- Το διατηρείτε στο ψυγείο

**Ultradent Products GmbH**  
**Am Westhover Berg 30**  
**51149 Cologne**  
**Germany**

**ULTRADENT**  
PRODUCTS, INC.

For an SDS of this product, please see: [www.ultradent.com](http://www.ultradent.com)

© Copyright 2016 Ultradent Products, Inc. All Rights Reserved. Made in USA

Manufactured by Ultradent Products, Inc. 505 West 10200 South, South Jordan, Utah 84095

051116 68581.5

# Opalescence 6% *go*<sup>®</sup>

- EN- Tooth Bleaching System
- DE- System zur Zahnaufhellung
- FR- Système pour blanchiment des dents
- NL- Tandbleeksysteem
- IT- Sistema per lo sbiancamento dei denti
- ES- Sistema de Blanqueamiento Dental
- PL- System wybielania zębów
- PT- Sistema de branqueamento dental
- SV- Tandblekningssystem
- DA- Tandblegningssystem
- FI- Hampaiden valkaisujärjestelmä
- EL- Σύστημα Λεύκανσης των Δοντιών

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'U' • Upper bleaching tray  
'L' • Lower bleaching tray  
Product must be refrigerated. Before using, allow product to warm to room temperature. This will make it easier for the tray to adhere to the teeth. Remove from package.

Center tray on arch.

Gently suck down on the tray.

Remove outer tray.

Suck down or swallow again.

Wear for 60-90 minutes per day. When finished remove bleaching tray and brush teeth.

## Description:

Opalescence Go® is a dentist-controlled, dentist-supervised, take-home bleaching system, dispensed in pre-loaded disposable trays. It is a clear, flavored, high viscosity, sticky, 6% hydrogen peroxide gel with potassium nitrate and sodium fluoride.

## Precautions for Europe:

1. Not to be used on a person under 18 years of age.
2. To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.
3. Contains sodium fluoride.
4. Please note the expiration date printed on the outer package/tray containers.

## Precautions:

1. Keep tooth bleaching product away from small children.
2. A dental examination is recommended prior to bleaching. Some discolorations may indicate the need for dental treatment other than bleaching. Porcelain crowns, veneers, and white fillings do not lighten upon bleaching.
3. Only intended for use by those with healthy gum tissue. Gum tissue irritation may result if used in individuals with periodontal disease
4. This product is NOT for nighttime use.
5. Occasionally, some parts of teeth may bleach faster or less evenly than others. Results may be varied. In approximately 5% of the population, teeth may be resistant to bleaching. Bleaching teeth with large metal fillings may cause the metal color to show through the teeth.
6. Acidic foods/juices may cause sensitivity if consumed shortly after bleaching.
7. Do not use tobacco products or eat while bleaching.
8. Not intended for use by pregnant women.
9. Keep gel out of heat/sunlight. Do not freeze.
10. Do not swallow gel or rinses. Product contains peroxide and may contain fluoride; swallowing large amounts can be harmful.
11. Materials should be refrigerated (2°-8° c) to maintain shelf life.
12. Avoid contact with the eyes.

**WARNING:** Some users may experience sensitivity, which if it occurs is most often mild and transient, but in some cases may require treatment by your dentist. If you experience significant discomfort, stop using this product and consult your dentist. If you have any questions about this product or treatment, please ask your dentist. Do not reuse to avoid cross contamination.

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