· Conscious dentistry ·

STEAM CONTROL STRIPS Type 4

en · Chemical integrator for steam sterilization control
es · Cinta indicadora química para la esterilización por vapor
it · Integratore chimico per il controllo della sterilizzazione a vapore
fr · Intégrateur chimique pour le contrôle de la stérilisation à la vapeur
pt · Integrador químico para controlo da esterilização a vapor

"THE FUTURE
IS IN YOUR HANDS"

VIDU

en

INSTRUCTIONS FOR USE

STEAM CONTROL STRIPS Type 4 are 1.7 cm wide by 20 cm long paper indicator strips printed with a chemical indicator ink that turns from yellow to black when exposed to the steam sterilization process.

The STEAM CONTROL STRIPS Type 4 is designed to monitor all vacuum and gravity assisted steam sterilization cycles at 121-134°C and indicate whether steam has penetrated to the strip placement point, typically the center of a package.

INDICATIONS FOR USE:

The STEAM CONTROL STRIPS Type 4 chemical indicator strip is used to monitor all gravity and vacuum assisted steam sterilization cycles at 121-134°C.

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Reference Color Change Conditions
Marks

Saturated steam,
132 °C, 3 mins
with pre-vacuum

UNPROCESSED

FAILURES

One or more sterilization parameters were not achieved. The possibility of unit sterilization failure is high, and the release of this unit should be determined based on the results of bi and risk assessment, and the sterilizer should be inspected.

The main failure causes are:
inadequate cycle duration
improper cycle selection
overloading / incorrect loading of the sterilizer
poor steam quality
sterilizer malfunction
blocking indicator with large metal mass

PROCESSED AND SATISFACTORY

PRECAUTIONS:

Do not use STEAM CONTROL STRIPS Type 4 steam chemical integrators to monitor dry heat, ethylene oxide, hydrogen peroxide, or other low temperature sterilization processes.

STORAGE/SHELF LIFE/DISPOSAL:

- Best stored at normal room temperature conditions, 15-30°C (59-86°F) and 40-60% relative humidity. Store away from direct sunlight. Do not store near strong alkaline or acidic products such as cleaning/disinfecting agents.
- After use, the indicator will not change visually within 6 months when stored at above conditions.
- 2 years from the date marked on the packaging

PACKAGING:

REF V115202 STEAM CONTROL STRIPS Type 4, 200 strips

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