# VOCO Structur Premium

Instructions for use

## Product description:

**Structur Premium** is a self-curing, highly aesthetic paste-paste system for the fabrication of temporary crowns, bridges, inlays and onlays. **Structur Premium** has excellent mechanical properties and is fluorescent. **Structur Premium** is available in 7 shades: A1, A2, A3, A3.5, B1, B3 and BL.

## Directions:

Insert **Structur Premium** cartridge into the **VOCO** Dispenser (type 2) or a compatible dispenser with a suitable piston. Due to technical reasons, there may be slight differences in the filling levels of the two cartridge chambers. In this case, remove cartridge cap and extract material until both chambers deliver uniformly. Then attach a mixing tip type 6 and lock with a 90° turn. By pressing the lever, the material is extracted and automatically mixed in the correct ratio.

The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by **VOCO**. All the mixing tips for use with our respective products are available from **VOCO**.

Do not remove the mixing tip from the cartridge after use. Only replace the used mixing tip with a new one immediately before the next application. When changing the mixing tip, check the opening of the cartridge to confirm that the material can flow freely.

## Advice on application:

First level out any undercuts in the preparation using a non-eugenol cement. Take an impression if prefabricated crowns or similar restorative pieces are not being used.

Prepare the impression by levelling undercuts and cutting drainage channels. Enlarge the impression if the wall thickness of the temporary restoration is expected to be too thin, taking care to remove the septums in the interdental embrasures. Clean and air-dry the impression afterwards.


Apply **Structur Premium** directly into the impression. Do not overfill.

The impression must be inserted into the mouth within 30 s. Remove the temporary from the mouth during the elastic phase, 1:00 min - 1:30 min after commencement of mixing.

The setting process can also be monitored by testing the excess material in the mouth.

The setting time of non-used material (in mixing tips) **cannot** be compared to the setting time of the material in the mouth.

At a room temperature of 23 °C and 50% humidity, working times are as follows:

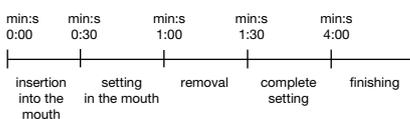Slight differences may result from different room temperatures.

After approx. 4 min, take the temporary out of the impression and wipe dry. If necessary, a solvent such as ethyl alcohol may be used. This removes the thin inhibition layer usually caused by oxygen and allows easier finishing. The surface of the temporary restoration should appear silky-matt.

Carefully trim any excess material and roughen areas near the gingiva. Lute the temporary restoration using an eugenol-free temporary cement.

## Information, precautionary measures:

- **Structur Premium** contains methacrylates, amines, terpenes, benzoyl peroxide and BHT. **Structur Premium** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
- Before constructing the temporary, prepared cores should be insulated to promote desensitization (fluoride varnishes, core varnishes, etc.).
- In order to avoid fractures or injuries caused by fractures, the temporary should not be subjected to excessive loads (molar areas).
- Composite-based linings and core build-ups should be sufficiently insulated to prevent them from being accidentally removed at the same time as the temporary (e.g. glycerine).
- Inadequate hygiene or frequent use of certain mouthrinses may lead to discoloration.
- Finishing of the temporary should start only after the material has set completely (approx. 4 min after commencement of mixing).
- Restoration of defective temporaries:
- If the temporary breaks despite the extreme stability of **Structur Premium**, the following procedures are recommended:
  - **Air bubbles:** fill with **Structur Premium**.
  - **In case of fracture immediately after construction of the temporary:** re-connect the fracture with freshly mixed **Structur Premium**.
  - **Repairing fractured temporaries that have already been worn:** Roughen the fracture with a bur and place undercuts. Apply bond to the prepared fractures and cure according to the respective instructions for use. Apply a flowable composite on the prepared areas, assemble the fractured pieces and cure according to the instructions for use.

## Storage:

Store at temperatures between 4 °C - 23 °C.

**Protect from bright light, i.e. store in closed packages, drawers, etc.**

Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 BB1712 E1 0917 99 © by VOCO
**VOCO GmbH**